## Instructions for use

For the MYND device to successfully operate the gels pads must be applied correctly. Improper application can result in uneven or ineffective stimulation.

- 1) Before use, it is essential to clean your skin with a damp cloth and soapy water. Leaving your forehead slightly damp may allow for better adhesion and stimulation from the device.

**i Note:** Do not use make-up removal products to clean your forehead as these leave a greasy film on the skin.

- 2) For first use, remove one of the gel pads from its foil packaging.
- 3) Remove the blue liner from the gel pads and apply to the rear of the device.
- 4) Remove the clear plastic liner from the device. Be careful to ensure the gel pads remain stuck to the device.

**i Note:** Please take care when removing both liners from the gel pad to ensure the gel does not get damaged.

- 5) Place the clear plastic liner in the travel case. The blue liner can be disposed of.
- 6) Place the device in the centre of your forehead, roughly 1 cm above the eyebrows.

**! Do not** place gel pads on skin which does not have normal sensation or on broken skin.

**! Do not** ignore any allergic reaction to the electrode pads: If a skin irritation develops, stop using MYND, as a small number of users may have an adverse reaction to the gel.

The unit will turn off automatically when treatment session is complete.

- 1) Remove the device from your forehead by peeling the device by the outer edges.
- 2) Re-apply the clear plastic liner on the gel pads and store the device in the travel case. This will help retain the stickiness of the gel.
- 3) Once the gel pad loses its stickiness, apply a new MYND gel pad to the device.

### Advice

The gel pads supplied are reusable but should be single patient use.

Replace the gel pad when they lose their stickiness or dry out.

The number of applications for each gel pad will vary depending on the user's skin condition and humidity. Each gel pad should provide 3 - 5 applications.

In very hot weather the gel on the pads may become soft.

Always check that the unit is OFF before applying or removing the gel pad.





### Storage Life

The storage life of an unopened pack of gel pads is 2 years. This may be affected by very high temperatures or very low humidity.

Keep gel pads in sealed plastic pouch when not being used. This will help maximise the storage life.

### Important

Observe all the Contraindications, Cautions and Warnings included with the device that you are using with these gel pads.

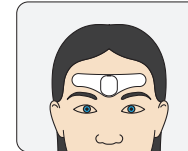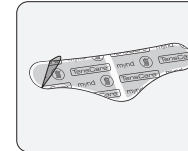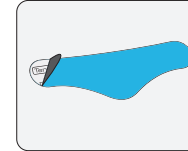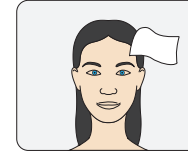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TensCare Ltd. 9 Blenheim Rd, Epsom, Surrey, KT19 9BE, UK, +44 (0)1372 723 434.
**EC REP** Advena Ltd. Tower Business Centre, 2nd Flr, Tower Street, Swatara, BKR 4013, Malta