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**REF** X-AP

## Small Anal or Vaginal probe for electrotherapy unit

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**INSTRUCTIONS FOR USE**  
READ CAREFULLY BEFORE USE

**TensCare™**

## INDICATIONS FOR USE

The **Liberty mini** probe is specifically designed to suit your body. The small dimensions, smooth outer casing and curved edges mean it is easy to insert and comfortable to use.

The probe has a curved tip for easy insertion, and a ring at the base for easy removal.

For use only with Muscle Stimulation and EMG Biofeedback equipment approved to EN 60601-1 in the treatment of Stress, Urge and Mixed urinary incontinence, faecal incontinence and pelvic pain.

**For Women:** The probe is designed for anal use but can also be used vaginally for women who prefer a smaller probe and for those suffering from vaginal atrophy. When used in the anus, the probe is an effective way to target both the rear part of the pelvic floor muscles if you suffer from rectocele (rectal prolapse) and also the anal sphincter muscles which helps to treat bowel incontinence and excessive flatulence.

**For Men:** The probe should be used in the anus.

Pelvic floor muscle training is recommended for recovery from some causes of Erectile Dysfunction. However, it is not intended to treat any medical issues and your first step in deciding appropriate therapy should be to consult your professional medical advisor.

## PROBE CAUTIONS

**Caution:** This probe is intended for single patient use only. Do not share your probe with anyone else. *Improper treatment or cross-infection may occur.*

**Caution:** This probe is intended for single orifice use only (intra-vaginal or intra-anal use). *Improper treatment or cross-infection may occur.*

**Caution:** It is important that the probe is cleaned before and after each use. *Ineffective cleaning may lead to irritation or infection.*

**Caution:** Never insert or remove the probe unless the control unit is powered OFF as insertion or removal when stimulation is active may cause discomfort or tissue irritation.

**Caution:** If tissue irritation occurs, discontinue treatment immediately. *Ask your healthcare professional for advice before continuing further treatment to prevent injury.*

**Caution:** Do not use a silicone based lubricant on the metal plates of the probe as it may decrease the effectiveness of the electrical stimulation.

**Caution:** The stainless steel in the probe's metal plates contain some Nickel. This could cause a reaction if you have a Nickel allergy. *Alternative gold probe specially made without Nickel is available (see X-VPG).*

**Caution:** Electrode surface area is 3.35cm<sup>2</sup>. At high output settings on some devices, this could result in a current density greater than 2mA RMS/cm<sup>2</sup>. High current density may be associated with localised tissue damage or minor non-thermal burns. Increase intensity carefully and reduce if you experience sharp pain during use or discharge or bleeding after use.

**Warning:** Do not use if the pack is opened or damaged.

## INSTRUCTIONS FOR USE

1. Connect the lead from the base of the unit to the lead in the probe. Push the pin ends firmly into the pigtail ends of the probe lead.
2. Before using the probe, you will need to visit the toilet.
3. Lubricate the metal electrode surfaces and probe tip with a water-based lubricant, such as TensCare Go Gel or water.

-  **Caution:** Do not use a silicone based lubricant on the stimulation contacts as it may decrease the effectiveness of the muscle stimulation.
4. Choose a comfortable position, such as lying down on your bed on your side with your knees raised.

 **Warning:** Ensure the unit is switched OFF before inserting the probe.

5. **a) Anal insertion:** After wires are securely connected, insert the probe into the anus whilst 'bearing down' (as in the action of passing stool) to a comfortable limit until the base of the flange on the probe touches the anus. The metal parts conduct the electrical pulse and should be in contact with the main part of the muscle at all times. The tissues close to the entrance are more sensitive, so you should avoid stimulating them. It is recommended that the probe is inserted past the sphincter muscles of the anus, unless directed otherwise by a healthcare professional. Anal probes with long electrodes (the metal part) that run up and down the length of the attachment should always be inserted with the metal parts facing hip-to-hip.

 **Note:** Sometimes the wearing of tight fitting undergarments or a tight pair of jeans will help to keep the probe in place and maintain correct contact during the programme.

**b) Vaginal insertion:** After the wire is securely connected, insert the probe into your vagina, in the same way as you would a tampon, until only the plastic ring at the base of the probe is visible. The metal parts conduct the electrical pulse and should be in contact with the main part of the muscle at all times. The tissues close to the entrance are more sensitive, so you should avoid stimulating them. Probes with long electrodes (the metal part) that run up and down the length of the attachment should always be inserted with the metal parts facing hip-to-hip.

6. After your training session, switch the control unit off, then remove the probe by holding the positioning end ring and gently pulling outwards.

## CLEANING

It is important that the probe is cleaned before and after each use. Clean with either an alcohol-free antibacterial wipe such as TensCare Wipes (see X-WIPES) or by wiping with warm soapy water. Rinse under running water and dry thoroughly and return the unit to the storage pouch.

### Warnings:

Do not immerse the probe in a liquid.

Keep the device away from lint and dust, as long-term exposure to lint or dust may affect the socket.

Keep the device away from sunlight, as long-term exposure to sunlight may affect the rubber causing it to become less elastic and crack.

## DISPOSAL

Clean thoroughly and dispose of according to local regulations.

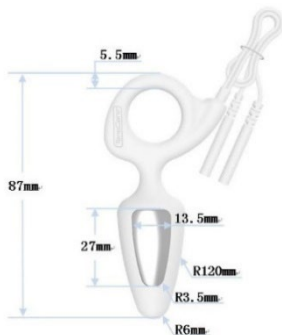
## TECHNICAL SPECIFICATIONS

Material: Electrode body: Plastic ABS PA-757

Electrodes: Stainless steel #304

Cables: PVC over copper tinsel conductor

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**TensCare Ltd,**
**Advena Ltd.**