Instructions For Use  
Please read carefully before use

REF E-BF155

# Electrodes

## Butterfly 105x155mm

- 2 Reusable butterfly electrode pads for electrotherapy unit
- 2 Electrodoos de mariposa reutilizables para dispositivo de electroterapia
- 2 Electrodes papillons réutilisables pour appareil d'électrothérapie
- 2 Wiederverwendbare Schmetterlingselektroden für elektrostimulation Gerät
- 2 Farfalla elettrodi riutilizzabili per unità di elettroterapia
- 2 Электрода многоразового применения для приборов электротерапии

TensCare™

## Instructions for Use

Use with a TENS or EMS unit such as TensCare **perfect TENS**, **perfect BEAUTY** or **Sports TENS**. For pain relief, use these electrode pads with TENS in the settings you would normally use. For toning, use these electrodes with an EMS programme for large muscles.

1) Before use make sure your skin is clean and dry.

**Do not place electrode pads** on skin, which does not have normal sensation or on broken skin.

2) Push the pin ends of the lead wires firmly into the pigtail ends of the electrode pads.

**Note:** A single electrode pad acts as a pair of TENS electrodes. Connect both pigtail leads to the same TENS lead.

3) Peel the electrode pads from their protective plastic shield by holding and lifting one corner of the pad and pulling.

**Do not** pull on the pigtail wire of the pad.

**Always check** that the unit is **OFF** before applying or removing electrode pads

4) Apply the adhesive side of the electrode pad around the area of pain\* and firmly press it to the skin.

5) Remove the electrode pads from your skin by holding the pad itself and gently pulling.

6) After use always replace the electrode pads on the plastic liner and replace them in the re-sealable plastic bag.

**Important:** Observe all the Contraindications, Cautions and Warnings included with the device that you are using with these electrodes.


## Advice

- The electrode pads supplied are reusable but for single patient use.
- The electrode pads provided are latex-free.
- To obtain the best conductivity through the electrode pads always ensure that they are in good condition and tacky.
- Replace the electrode pads when they lose their stickiness or dry out.


- In very hot weather the gel on the electrode pads may become soft.


\* **Electrode pads placement:** The butterfly electrode pads can be used on any large areas of pain. The most common uses are shown below.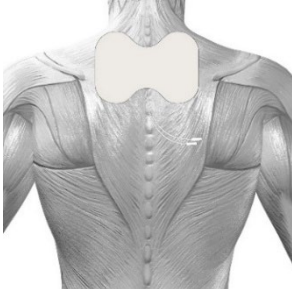


**Upper back pain**

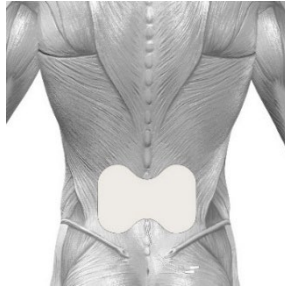**Lower back pain or Sciatica (option 1)**

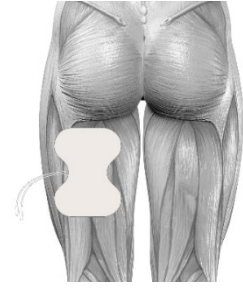**Sciatica (option 2)**

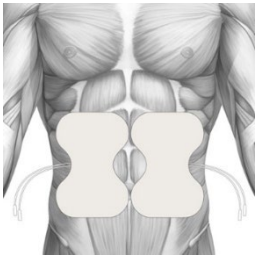**Abs toning**

**Bottom toning**

## Storage Life

- Electrode pads should last 12 to 20 applications, depending on skin condition and humidity.
- Storage life of an unopened pack of electrode pads is 4 years. This may be affected by very high temperatures or very low humidity.
- Surface area: 54 cm<sup>2</sup>

## Warnings

- Do not ignore any allergic reaction to the electrode pads: If a skin irritation develops, stop using device, as a small number of users may have an adverse reaction to the gel.
- For use only with Nerve Stimulators approved to EN 60601-1 and EN 60601-2-10.
- The electrode pads supplied are reusable but for single patient use. The adhesive is a peelable hydrogel (water based).

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**TensCare Ltd.**
**Advena Ltd.**