
Electrodes 50x100mm

4 Reuseable Electrode Pads

REF E-CM50100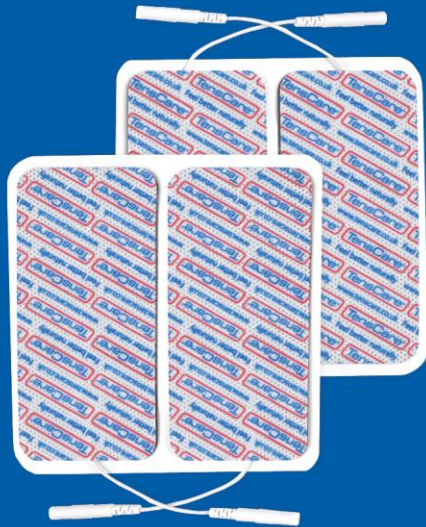


Instructions For Use
Please read carefully before use

TensCare™

TensCare™

I-CM50100-MLS Rev 4.2 03/24

www.tenscare.co.uk

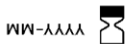YYYY-MM

XXXXXX

LOT

TensCare Europe Bv
Kasselaarweg 18, 1704 DW,
Neerhugowaard,
Netherlands

500µPa
-1060µPa

Storage & Transport
-5°C - +40°C

20% RH
60% RH


Adventa Ltd.
Tower Business Centre,
2nd Flr, Tower Street,
Swatara, BKR 4013, Malta

TensCare Ltd.
9 Blenheim Rd, Epsom,
Surrey, KT19 9BE, UK
+44 (0)1372 723 434

UDI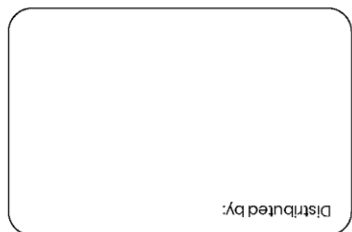

Distributed by:

Instructions for use

For the desired success of any stimulation (TENS/EMS), it is important that electrode pads are placed sensibly. Leave 5-15 cm between the 2 electrode pads on a channel.

i Note: The electrode pads must always be used in pairs (two electrode pads on each channel), so that the signal can flow in a circuit.

1) Before use make sure your skin is clean and dry.

! **Do not place electrode pads** on skin, which does not have normal sensation or on broken skin.

2) Push the pin ends of the lead wires firmly into the pigtail ends of the electrode pads.
3) Peel the electrode pads from their protective plastic shield by holding and lifting one corner of the pad and pulling.

! **Do not pull** on the pigtail wire of the pad.

Always check that the unit is **OFF** before applying or removing electrode pads.

4) Apply the adhesive side of the electrode pads around the area of pain and firmly press it to the skin. See examples of positioning in the placement guide enclosed.

5) Remove the electrode pads from your skin by holding the pad itself and gently pulling.
6) After use always replace the electrode pads on the plastic liner and replace them in the re-sealable plastic bag.

Advice

- The electrode pads supplied are reusable but for single patient use.
- The electrode pads provided are latex-free.
- To obtain the best conductivity through the electrode pads always ensure that they are in good condition and tacky.
- Replace the electrode pads when they lose their stickiness or dry out.
- In very hot weather the gel on the electrode pads may become soft.

Storage life

Electrode pads should last 12 to 20 applications, depending on skin condition and humidity.

Storage life of an unopened pack of electrode pads is 2 years. This may be affected by very high temperatures or very low humidity.

Warnings

Do not ignore any allergic reaction to the electrode pads: If a skin irritation develops, stop using device, as a small number of users may have an adverse reaction to the gel.

For use only with Nerve Stimulators approved to EN 60601-1 and EN60601-2-10

Important: Observe all the Contraindications, Cautions and Warnings included with the device that you are using with these electrodes.

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