



Electrodes

Boots Press Stud Electrodes

4 Reuseable Electrode Pads

For single patient use

REF E-ST



Instructions For Use

Please read carefully before use

TensCare™



I-EST-UK Rev 3.1/09/21



TensCare Ltd.
9 Blenheim Rd, Epsom,
Surrey, KT19 9BE, UK
+44 (0)1372 723 434
www.tenscare.co.uk

Advena Ltd.
Tower Business Centre,
2nd Flr, Tower Street,
Swatar, BKR 4013, Malta

TensCare Europe BV
Juliana van Stolbergstraat 52,
1723LD Noord-Scharwoude,
Netherlands



For use only with Nerve Stimulators approved to EN 60601-1 and EN60601-2-10

Do not ignore any allergic reaction to the electrode pads: if a skin irritation develops, stop using device, as a small number of users may have an adverse reaction to the gel.

Warnings

1. Between uses keep the electrodes on the liners and store in the re-sealable bag in a cool place, out of direct sunlight.
2. When the electrode gel appears dry apply a few drops of water, spread with fingertip and let rest until gel regains its tackiness.
3. If electrode becomes over-saturated, allow to air dry in a refrigerator, adhesive side up, until gel regains its tackiness.

Storage and Maintenance

Turn off stimulation device and disconnect wires. Remove electrodes by peeling from an edge.

Removal of electrodes

6. Attach the lead wires to the electrodes and the stimulation device.
with your medical adviser.
5. Place electrode on skin by applying the centre of the electrode first then smoothing outwards to the edge. Use the illustrations in the instruction manual as a guide to positioning or consult Note: Retain liner(s) for storage of electrodes after use.
4. Remove an electrode from its liner ONLY by holding one edge and peeling it off the liner.
3. Connect lead wires to the electrodes by pressing the metal studs together.
2. Clean electrode application area with soap and water, then rinse and dry.
1. Make sure stimulation device is switched off prior to applying or removing electrodes.

Application of electrodes