




Electrodes

Boots Press Stud Electrodes

4 Reuseable Electrode Pads

For single patient use

REF E-ST

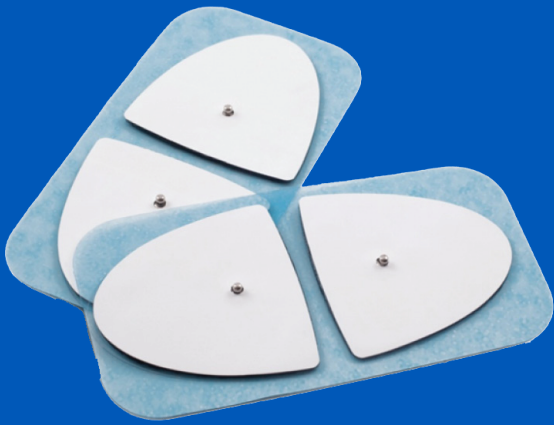

Instructions For Use

Please read carefully before use



1-EST-UK Rev 3.2 10/24



Tower Business Centre, 2nd Flr, Tower Street,
Swatar, BKR 4013, Malta

Advena Ltd.



Beukenlaan 23-033, 1701 DA,
Heerhugoward, Netherlands

TensCare Europe BV



9 Blenheim Rd, Epsom, Surrey, KT19 9BE, UK
+44 (0)1372 723 434, www.tenscare.co.uk

TensCare Ltd.



Turn off device and disconnect wires. Remove electrodes by peeling from an edge.

Removal of electrodes

6. Attach the lead wires to the electrodes and consult with your medical adviser.

5. Place electrode on skin by applying the centre of the electrode first then smoothing outwards to the edge. Use the illustrations in the instruction manual as a guide to positioning or

4. Remove an electrode from its liner ONLY by pressing the metal studs together.

3. Connect lead wires to the electrodes by pressing the metal studs together.

2. Clean electrode application area with soap and water, then rinse and dry.

1. Make sure stimulation device is switched off prior to applying or removing electrodes.

Application of electrodes

1. Between uses keep the electrodes on the liners and store in the re-sealable bag in a cool place, out of direct sunlight.

Storage and Maintenance

Note: The life of the electrode depends on skin condition, preparation, storage and climate.

Do not ignore any allergic reaction to the electrode pads: if a skin irritation develops, stop using device, as a small number of users may have an adverse reaction to the gel.

3. If electrode becomes over-saturated, allow to rest until gel regains its tackiness.

2. When the electrode gel appears dry apply a few drops of water, spread with fingertip and let rest until gel regains its tackiness.

1. Between uses keep the electrodes on the liners and store in the re-sealable bag in a cool place, out of direct sunlight.

2. Clean electrode application area with soap and water, then rinse and dry.

3. Connect lead wires to the electrodes by pressing the metal studs together.

4. Remove an electrode from its liner ONLY by pressing the metal studs together.



For use only with Nerve Stimulators approved to EN 60601-1 and EN60601-2-10