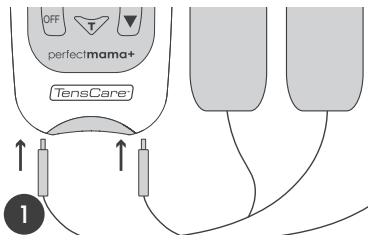
perfectmama+


Instructions for Use
Please read carefully

TensCare™


Quickstart Guide


1 Connect the device to the electrodes.

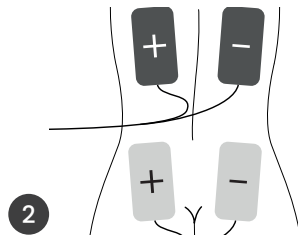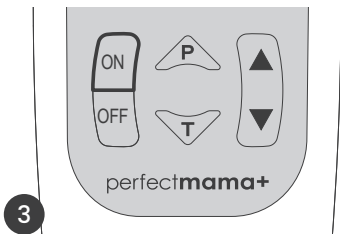
2 Place the electrodes on the area to be treated.

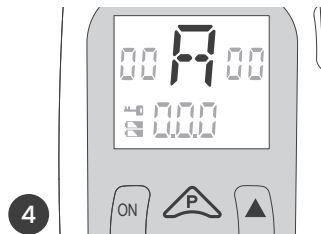
3 Press and hold the power button to turn the device on.

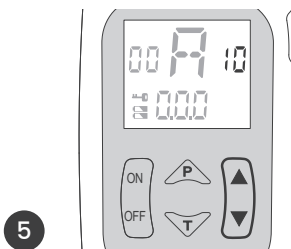4 Select a programme by pressing the P button.

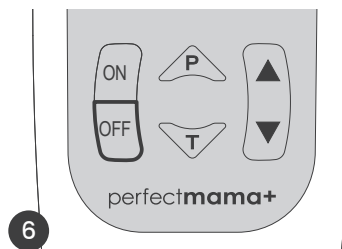5 Start the session by increasing the intensity.

6 Press and hold the power button to turn the device off.

Contents

1. Device Description	3
1.1. Introduction	3
1.2. Intended purpose	3
1.3. Kit contents	3
1.4. Controls and display	4
2. How to Use	4
2.1. Device setup	4
2.2. Providing relief during labour	6
2.3. After labour	9
2.4. Alternative uses	10
3. Troubleshooting	12
4. Device Care	13
4.1. Cleaning	13
4.2. Storage	13
4.3. Battery replacement	13
4.4. Electrode and lead wire care	14
4.5. Replacement parts/accessories	14
5. Technical Specification	15

1. Device Description

1.1. Introduction

The Perfect mama+ is a clinically proven, electrotherapy stimulator that uses Transcutaneous Electrical Nerve Stimulation (TENS) to provide drug-free pain relief during childbirth.

The device delivers small electrical signals to the body via body-worn, adhesive electrodes. These signals stimulate the nerves, helping to block pain signals and promote the release of endorphins, the body's natural painkiller.

Perfect mama+ may also be used to relieve back pain during the last three weeks of pregnancy and to ease post-operative pain following caesarean, perineal tear or episiotomy.

When used as intended, the Perfect mama+ can provide the following clinical benefits:

- Reduction in pain during labour
- Reduction in post-operative pain
- Reduction in back pain during the last three weeks of pregnancy

Note: While some expectant mothers find the Perfect mama+ provides sufficient pain relief on its own, others may need additional relief. The Perfect mama+ can be effectively combined with other pain relief options, such as gas and air. However, due to its electrical nature, it cannot be used during water births.

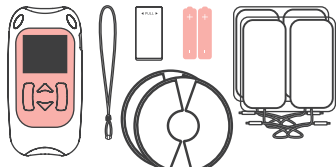
1.2. Intended purpose

Perfect mama+ is designed to be used in the home healthcare and hospital environments to provide symptomatic relief and management of acute pain in the perinatal* and postpartum period and is suitable for use by all who can control the device and understand the instructions. TENS could also be used to relieve back pain in the last 3 weeks of pregnancy and to ease post-operative pain.

Do not use the device for any purpose other than this intended use.

i Note: Perinatal is the period immediately before and after birth. The perinatal period is defined in diverse ways. Depending on the definition, it starts at the 20th to 28th week of gestation and ends 1 to 4 weeks after birth.

1.3. Kit contents


- 1x Perfect mama+ device
- 4 x 50x100mm electrodes (E-CM50100-PM)
- 4 x Breast electrode pads (E-CMR60130)
- 2 x Lead wires (L-CPT)
- 1 x Detachable belt clip
- 2 x AA batteries

1 x Neck cord

1 x Storage pouch

1 x Instructions for Use

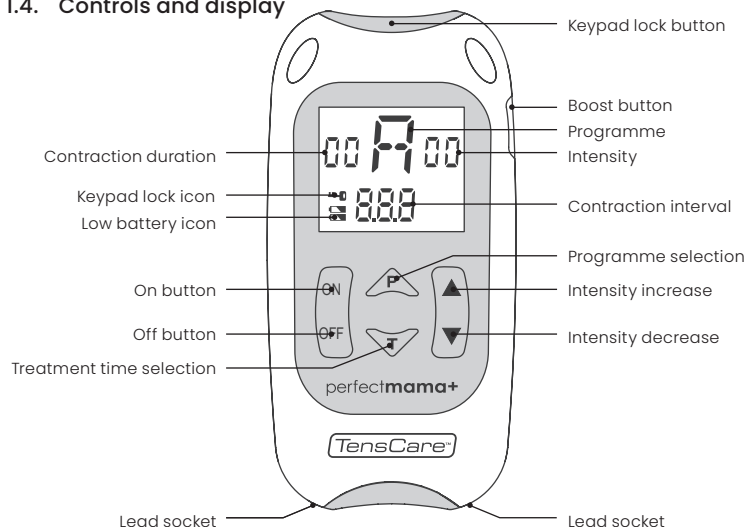1.4. Controls and display



2. How to Use

2.1. Device setup

Install the batteries

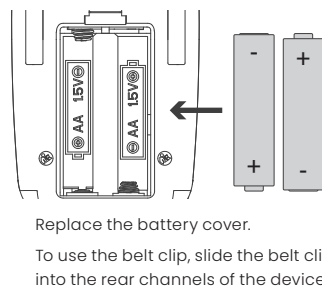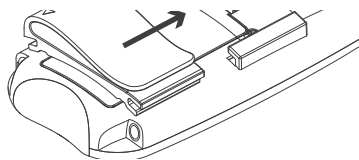1. Remove the battery cover on the rear of the device.
2. Insert the batteries, ensuring that the positive (+) and negative (-) ends match the corresponding terminals in the compartment.


5. To use the neck cord, loop the cord through one of the holes on the top of the device.


Connect the leads

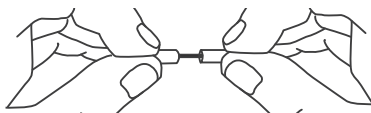1. Insert the white end of the lead into the channel socket on the rear of the device.
2. Push the metal pins of the lead wires into the sockets on the electrodes.


Position the Electrodes

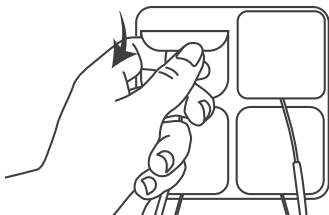1. Before applying the electrodes, make sure your skin is clean and dry
2. Remove the electrodes from the plastic liner.

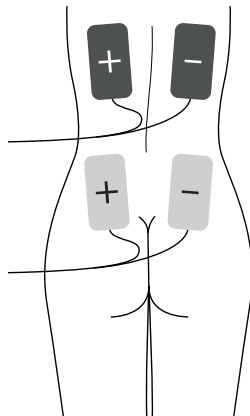3. Position the two electrodes from the left-hand socket on either side of the spinal column, 5 cm apart and about 7.5 cm below the bottom of the shoulder blades. This area is just below the bra line.
4. Position the two electrodes connected to the right-hand channel on either side of the spine, approximately 10 cm apart and 2.5 cm above the dimples at the base of the spine, just above the buttocks.

Note: Always ensure the device is turned off or the intensity is zero before applying the electrodes.

Turning on / off

Turn the device on by pressing and holding the on button for 3 to 5 seconds.

Turn the device off by pressing and holding the off button for 3 to 5 seconds.

Note: The device will automatically turn off if the intensity setting is left at zero for more than 5 minutes.

2.2. Providing relief during labour

Start of labour

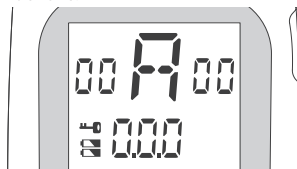
Perfect mama+ features three distinct programmes tailored for different stages of labour.

Begin using the Perfect mama+ when your contractions become regular.

Start with programme 'A'. If 'A' is not already selected, press the 'P' button to switch between programs. The screen will display a flashing 'A'.


Programme 'A' should be used in between contractions.

Intensity selection

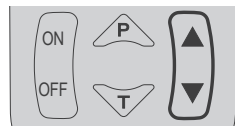Increase the intensity by pressing the ▲ button.

In programme 'A' there are 50 levels of intensity.

Each press of the ▲ and ▼ buttons will adjust the intensity by 1mA. The intensity can be increased or decreased quickly by pressing and holding the ▲▼ buttons.

The stimulation will produce a pulsing sensation. At low intensity levels, it is common for no sensation to be felt. Adjust the intensity until the feeling is strong yet comfortable.

Further increases during use may be necessary if your body becomes used to the sensation or as the pain from contractions intensify.

Note: You may notice that the upper electrodes provide stronger stimulation than the lower ones. This is intentional and part of the therapy. If you feel stronger stimulation at the lower electrodes, switch the leads in the device's sockets.

Note: Intensity cannot be increased if the device has not been correctly connected to the body (i.e. a closed circuit is made). If the Perfect mama+ detects a disconnection in either channel it will automatically return the strength to zero and the CH1, CH2 and 00 will flash. To increase intensity, ensure the electrodes are

correctly connected to the body and the device.

Warning: When the yellow LEDs in the channel output sockets light, this indicates the stimulation intensity is high enough to be harmful if not used correctly. Please use with caution.

During contractions

During contractions, when the pain intensifies, the stimulation can be boosted.

Press the BOOST button on the side of the device at the start of a contraction. The screen will display the letter 'A' constantly and will switch to 'A BOOST' mode.

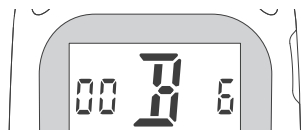
This changes the stimulation from pulsing to constant, helping to relieve from the increased pain during contractions.

At the end of the contraction, press the BOOST button again to return to programme 'A'.

When contractions intensify and become more frequent

Use programmes 'A' and 'A BOOST' for as long as possible.

When contractions intensify and become more frequent, switch to programme 'B' by pressing the P button.

Programme 'B' uses a stimulation pattern which is designed to block heightened pain. As before, use this programme in between contractions.

The intensity will be reduced by 50% to avoid any sudden increases in stimulation that could cause discomfort.

Increase the intensity using the ▲ button until the sensation is strong yet comfortable.

In programme 'B' there are 50 levels of intensity.

As the contractions become more intense and closer together, increase the intensity further.

Again, to boost stimulation during contractions, press the BOOST button when the contraction starts to switch to program 'B BOOST'. The screen will display the letter 'B' constantly.

When the contraction ends, press the BOOST button again to go back to program B.

For the latest stages of pregnancy

When pain intensifies further and programmes A and B are insufficient, press the P button to switch to programme C. This has been designed to block the highest levels of pain.

Again, when changing programme the intensity will be reduced by 50% to avoid any sudden increases in stimulation that could cause discomfort.

Increase the intensity using the ▲ button until the sensation is strong yet comfortable. In programme C there are 60 levels of intensity.

In this mode, when the contractions start press and hold the BOOST button. This will increase the intensity until the BOOST button is released, providing additional brief, intense pain relief.

Note: Release the BOOST button if the stimulation is too intense and causes pain.

Once the BOOST button is released the intensity will return to the previous setting before the BOOST button was pressed.

As the intensity decreases, you can press and hold the BOOST button again to maintain the intensity at that level until the end of your contraction

Contraction timer

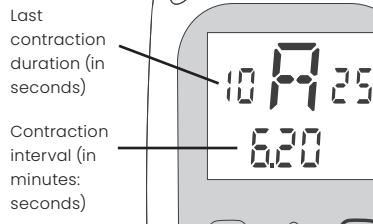
Perfect mama+ displays information about your contractions to help you and your midwife to understand how labour is progressing.

During contractions, the screen will

automatically display the duration of the last contraction and the interval between the last two contractions.

To turn the contraction timer on hold down the T and ▲ buttons for 3 to 5 seconds.

To turn the contraction timer off hold down the T and ▼ buttons for 3 to 5 seconds.

Contraction duration

Shows the duration in of time seconds of the last contraction.

The timer starts when you press the BOOST button to switch to constant mode (A BOOST or B BOOST).

The timer stops when you press the BOOST button again.

In programme C, the timer starts when you press and hold the BOOST button and stops when the unit returns to the intensity selected initially.

If the contraction lasts longer than 30 seconds, the display will flash for 30 minutes to indicate active labour has started.

Contraction interval

Shows the duration of time in minutes and

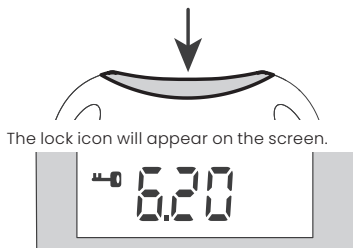
seconds between the start of your last two contractions.

If the contractions are less than 5 minutes apart, the display will flash for 30 minutes to indicate active labour has started.

Keypad lock

The keypad can be locked to avoid accidental changes in settings.

Press and hold the key button for three seconds to lock the device.


Press and hold the key button again to unlock the device.

Note: The intensity can be reduced by pressing the ▼ button whilst the device is locked.

Low battery

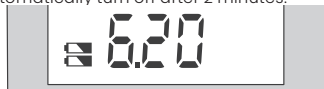When the battery is low, the battery icons will display on the screen. The device will automatically turn off after 2 minutes.

Replace with new AA batteries.

2.3. After labour

Disconnecting the device

After labour and relief is no longer required, turn the device off by pressing the Off button for 3 to 5 seconds.

Remove the electrodes from the skin by holding the pad and gently pulling. Do not pull on the lead wires.

Replace the electrodes on the plastic liner.

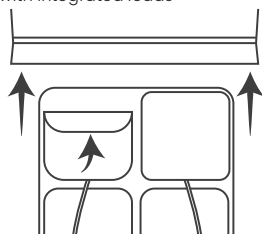
Disconnect the leads from the device.

Place the electrodes in the resealable plastic bag.

Return the device and electrodes to the storage pouch.

If you plan to put your Perfect mama+ into storage for your next baby then do the following:

- Remove batteries
- Clean as instructed in section 4.1
- Store or throw away the electrodes with integrated leads


- Remember to buy new electrodes and batteries before you need to use your Perfect mama+ again.

Time in labour

To see how long you have been using your Perfect mama+ press the T button.

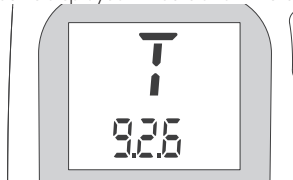The letter T will appear on the screen. Time in labour is displayed in hours and minutes.

After 3 seconds the screen will revert to normal.

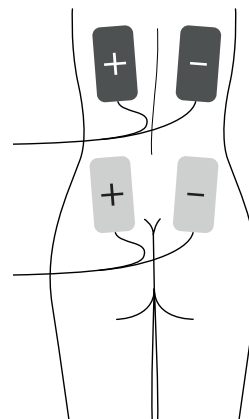
To manually reset time in labour to zero, press and hold down the T button for 5 seconds. The time automatically resets to zero when the Perfect mama+ has been turned off for more than 2 hours.

2.4. Alternative uses

Back pain before labour

Perfect mama+ can be used in the last three weeks of pregnancy to provide relief from back pain.

Apply the electrodes around the area of pain, with the electrode pairs positioned as follows:


Start with programme A and uses different programmes and/or BOOST modes if further relief is required.

Post-operative pain following childbirth

TENS stimulation can be used to help relieve post-operative pain.

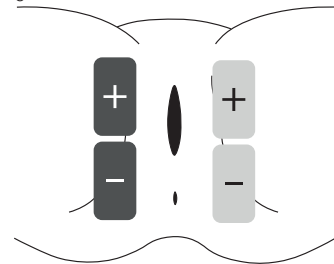Position the electrodes either side of the vagina, as follows:

Start with programme A and use different programmes and/or BOOST modes if further relief is required.

Warning: Before using Perfect mama+ for this purpose, please consult with your midwife or doctor.

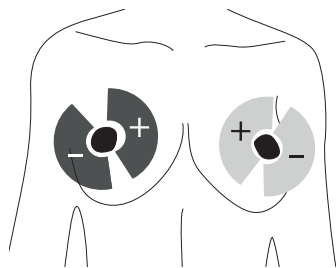
It's important to exercise caution when using TENS after surgical procedures as stimulation may interfere with the healing process.

For initiating lactation

Programme 'A' can be used with breast electrodes (E-CMR60130) has the potential to initiate lactation.

Place one pair of electrodes on each breast, 1-2 cm away from the areola, on opposite sides of the nipple.

Connect both electrodes on each breast to the same TENS channel (one to the red connector, the other to the black).

Use for 15 minutes every two hours.

TENS can stimulate the sensory nerves in the breast, mimicking a baby's suckling (over 100 times per minute). This can help

your body produce Prolactin, the hormone needed for lactation. You should feel a mild tingling during use.

Once milk production begins, do not use TENS as a replacement for regular pumping or breastfeeding. Use TENS as additional stimulation to support lactation, for example at night or during extra pumping sessions.

Warning: These pads are not intended to treat any medical issues, please consult with your midwife or doctor.

3. Troubleshooting

If the device is not working correctly, use the following troubleshooting guide.

Problem	Possible causes	Solution
Display will not turn on	Flat batteries.	Replace batteries.
	Batteries inserted incorrectly.	Remove plastic wrap. Check the positive and negative ends are positioned correctly.
	Damaged springs in battery compartment.	Contact supplier.
Controls will not work	The keypad is locked.	If the key icon is shown, press and hold the lock button for three seconds.
Low battery display	Low batteries.	Replace batteries.
No sensation	Intensity is not strong enough.	Increase the intensity. Most users will feel something at a setting below 10 mA.
Output will not increase above zero	There is a connection error leading to an open circuit.	Ensure machine is attached to your body correctly.
	Electrodes are not connected to body.	Purchase replacement electrodes if necessary.
	Leads are faulty/damaged.	
Sudden change in sensation	If you stop the stimulation and continue a few minutes later, the signal will feel quite a lot stronger.	Always return the intensity to zero after disconnecting the leads or electrode pads.

If you are unable to resolve the issue, please contact our customer service team:

Email: customer.service@tenscare.co.uk

Phone: +44 1372 723434

Note: The Perfect mama+ has no essential performance characteristics. Device failure will not

cause unacceptable clinical risk for the user.

Note: If the user is able to turn the device on, increase and decrease the intensity, switch between all three programmes and activate Boost mode in all programmes, this confirms the device will perform as intended by the manufacturer.

4. Device Care

4.1. Cleaning

Clean the device and lead wires at least once a week by wiping with a damp cloth and solution of mild soap and water. Wipe dry.

- Do not immerse the device in water.
- Do not use any other cleaning solution than soap and water.

4.2. Storage

Between uses it is important to store the electrodes in their resealable plastic bag with the bag sealed. This will help extend the life of the electrodes.

Keep the device and all accessories in the storage pouch between uses.

Storage life

- Storage life of an unopened pack of self-adhesive electrode pads is 4 years. This may be affected by very high temperatures or very low humidity.
- The device has no fixed shelf life.
- Unused batteries have a nominal shelf life of 3 years but will usually last longer than this.

4.3. Battery replacement

When the batteries are low and the battery icon is on the display, replace the batteries.

Batteries should last at least 30 hours when

using programme 'A' at 40mA.

The unit will work with rechargeable batteries, but the display may appear dim.

Battery Warnings

- Remove batteries from the device if it is unlikely to be used for a long period. Some types of batteries may leak corrosive fluid.
- Do not pierce, open, disassemble, or use in a humid and/or corrosive environment.
- Do not expose to temperatures over 60°C (140°F).
- Do not put, store or leave near sources of heat, in direct strong sunlight, in a high temperature location, in a pressurized container or in a microwave oven.
- Do not immerse in water or get wet.
- Do not short-circuit.
- Do not connect the device unless the battery cover is in place.
- If battery leakage occurs and comes in contact with the skin or eyes, wash thoroughly with lots of water and immediately seek medical attention.
- Keep batteries out of the reach of children to prevent them from swallowing them by mistake. If swallowed by child, contact doctor immediately.
- Never attempt to recharge an alkaline battery. Risk of explosion.
- Do not mix old, new or different types

of batteries as this may lead to battery leakage or low battery indication.

- Disposal: Always dispose of batteries and device responsibly according to local government guidelines. Do not throw batteries onto a fire. Risk of explosion.

4.4. Electrode and lead wire care

- The electrodes are reusable but for single patient use. The electrodes should provide up to 20 uses. This may vary depending on skin condition, humidity and storage between uses.
- To maximise the life of the electrodes, always ensure the electrodes are replaced on to their plastic liner and stored and sealed in the plastic bag. When the electrodes dry, this may negatively impact the performance of the treatment.
- The adhesive is the electrode is a peelable hydrogel (water based). To restore the tackiness of the electrode, add a tiny drop of water and spread it around. Avoid adding too much, as it will make the electrode too soft. If this occurs, place the electrodes sticky side up in a refrigerator for a few hours to regain adhesiveness.
- Replace the electrodes when they lose their stickiness. Poor connection may cause discomfort and skin irritation.
- To ensure safety and effectiveness we recommend using TensCare electrodes only.

- The life of the lead wires depends greatly on use. Always handle the leads with care. We recommend replacing the lead wires regularly (about every 6 months).

4.5. Replacement parts/ accessories


The following replacement parts are compatible with the Perfect mama+ and may be ordered from TensCare at www.tenscare.co.uk or +44(0) 1372 723434.

E-CM50100-PM	50x100 mm electrodes with integrated leads (pack of 4)
E-CMR60130	Breast electrodes (pack of 2 pairs)
L-CPT	Lead wire
B-AA	AA batteries (pack of 2)

5. Technical Specification

Waveform	Symmetrical Bi-Phasic
Amplitude (over 1 kOhm load)	75.0 mA zero to peak, 50 steps in A & B, 60 steps in C +/- 10%
Frequency range	80-150Hz
Pulse width range	150-250us
Output plug	Fully shielded: touch proof
Channels	Dual channel
Power supply	2 x AA alkaline (two AA batteries)
Weight	75g (without batteries)
Dimensions	120 x 60 x 20 mm
Safety Classification	Internal power source. Designed for continuous use. No special moisture protection.
Service life	>5 years
Applied part	Skin surface electrodes
Applied part classification	Type BF
IP rating	IP22
Environmental Specifications	
Operating conditions	Temperature range: 5 to 40°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa
Storage conditions	Temperature range: -25 to +60°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa

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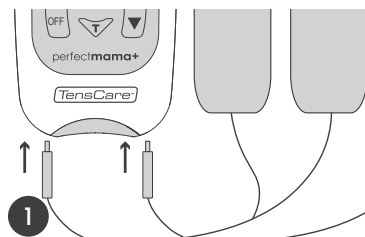Thank you for using Perfect mama+...

At TensCare, we strive to provide the best possible products and services. We value your feedback and continuously work to enhance our offerings. Understanding how our products benefit you is important to us. If you have any insights or suggestions, please get in touch.

Contact us

TensCare Ltd.
9 Blenheim Road,
Epsom, Surrey,
KT19 9BE, U.K.

+44 (0) 1372 723434

www.tenscare.co.uk

customer.service@tenscare.co.uk

Follow us

TensCare Ltd

@tenscareltd

@TensCareLtd

TensCare Limited

EU Declaration of Conformity

Tenscare Ltd herewith declare that the product meets the provisions of REGULATION (EU) 2017/745 which apply to it. The medical device has been assigned to class IIa according to Annex IV of the REGULATION (EU) 2017/745. The product concerned has been designed and manufactured under a quality management system according to Annex IX, of REGULATION (EU) 2017/745. This EU declaration of conformity is issued under the sole responsibility of the manufacturer. No "Common Specification" is applicable.