nouri duo

User Manual
Manual del usuario
Mode d'emploi
Benutzerhandbuch
Manuale d'uso

TensCare™

Contents

Introduction	3
Kit Contents	3
Pump Parts	3
Cleaning and Sterilising	4
Installation Guide	5
Pump Controls and Display	6
Mode Settings	7
Pumping with Nouri Duo	8
Storing Breast Milk	10
Preparing Breast Milk	10
Feeding Breast Milk	11
Charging your Nouri Duo	11
When to start using Nouri	11
Troubleshooting	12
Warnings	12
Technical Information	13
Warranty	14


Introduction

Nouri Duo is an automatic breast pump, designed to provide optimal comfort and performance whilst you express milk. The dual-pump allows you to express milk from both breasts simultaneously whilst the soft massage cushions gently stimulate milk flow. Eight suction levels ensure comfortable and quick pumping, so that mothers can enjoy every expression. The integrated LCD screen clearly displays suction level, working mode and battery level. Nouri Duo has a built-in rechargeable battery and integrated lights, allowing you to pump day and night in complete comfort.

Kit Contents

1 x Duo electric pump	2 x Bottle cap
2 x Bottle (150ml)	2 x Diaphragm
2 x Feeding teat and cap	4 x Bottle valve
2 x Diaphragm cap	2 x Tube
2 x Shield connector	1 x Bottle dock
2 x Breast shield	1 x USB cable
2 x Shield cover	

Pump Parts

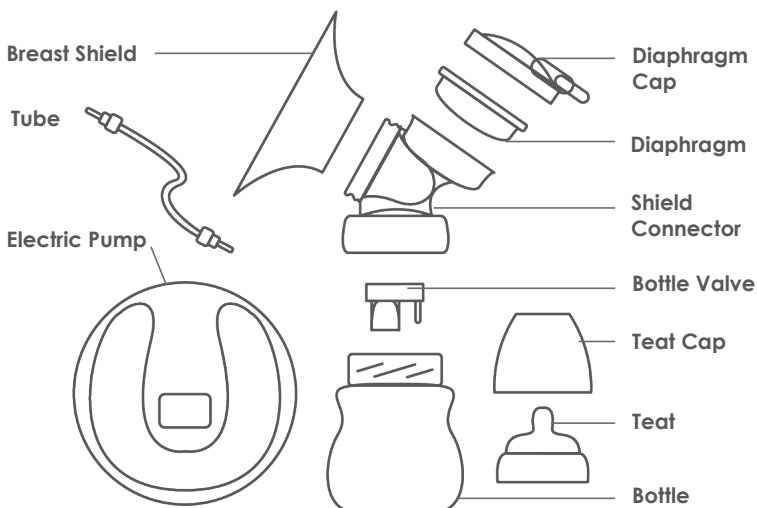Cleaning and Sterilising

Before using for the first time

Clean and sanitise the below parts before using your Nouri Duo for the first time. Use the following cleaning and sanitising instructions.

- Breast shield
- Shield connector
- Bottle valve
- Bottle

After each use

Clean all parts that come in contact with your breast and breast milk.

- Separate all parts, remembering to remove the bottle valve from the shield connector.
- Rinse the parts in cool water to remove breast milk residue.

Washing in sink

- Soak all parts in warm soapy water for 5 minutes and then clean with a clean dish-cloth or soft brush.
- Rinse all parts that come in contact with your breast and breast milk with clear water.
- Allow parts to dry.

Washing in dishwasher

- Wash all separated parts on the top rack of the dishwasher.
- Allow all parts to dry.

Sanitise daily

- Wash hands thoroughly.
- Separate all parts that come in contact with your breast and breast milk, remembering to remove the bottle valve from the shield connector.
- Immerse parts in boiling water for 10 minutes using a household pot.
- Allow water to cool before removing parts from the water.
- Place parts on a clean surface and/or towel and allow to dry.

Tubing and Electric Pump Unit

- Turn off the breast pump.
- Unplug the pump from any power source.

- Separate the tube from the pump and bottle.
- Wipe the electric pump unit and tube with a clean, damp cloth.

Caution:

Only use drinking tap water or bottled water for cleaning. Separate and wash all parts that come in contact with the breast and milk immediately after use to avoid milk residue drying and to prevent bacteria growth.

Do not store wet or damp parts.

During sanitising with boiling water, prevent the parts from touching the side of the pot. This can cause the parts to deform or be damaged. Nouri is for single person use. To avoid contamination, please do not share the pump.

Note:

Wash the Nouri Duo parts that come in contact with the breast and breast milk after every use. Sanitise the Nouri Duo parts that come in contact with the breast and breast milk once a day. Be careful not to damage parts during cleaning.

Installation Guide


1. Wash your hands thoroughly with soap and water.

2. Insert diaphragm into the connector and make sure the edge is engaged.


3. Put diaphragm cap on the connector and align the triangle marks on both components. The angle should be roughly 35 degrees.

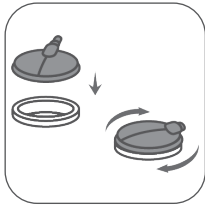4. Screw the diaphragm cap clockwise.


5. Engage the breast shield with the spout of the connector and make sure they are fixed securely.

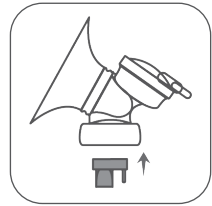6. Insert the valve into the connector and make sure they are fixed securely.


7. Screw the bottle into the shield connector assembly.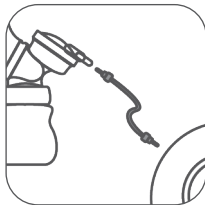

8. Plug one side of the tubing into the diaphragm cap. Connect the other side of the tubing with the airhole plug on the pump unit.

Pump Controls and Display

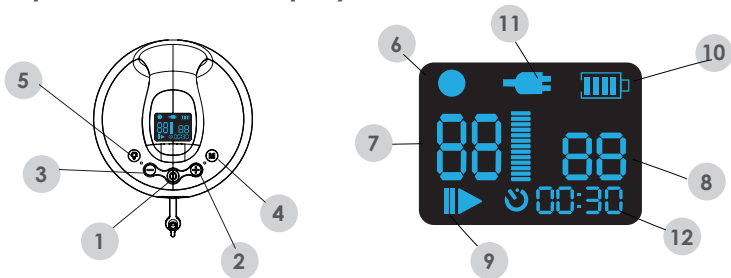1. ON/OFF

2. **Suction Increase:** Increase suction force

- 3. Suction Decrease:** Reduce suction force
- 4. Mode Switch:** Switch between massage mode and expression mode.
- 5. Night Lamp:** Press to turn on/off the lamp. Press once for low brightness, twice for high brightness.
- 6. Massage mode indicator:** Indicates the mode
- 7. Frequency:** Shows pumping frequency. Frequency is adjusted by pressing and holding the mode switch button until the frequency indicator flashes. Use increase and decrease to select pumping frequency.
- 8. Suction level:** Shows suction levels. Use increase and decrease to select suction level.
- 9. Pause/work indicator**
- 10. Battery level indicator:** Flashing icon means battery is low.
- 11. Power indicator:** indicates the power adapter is connected and the pump is being charged.
- 12. Time of working:** shows how long the pump has been used since turning on (default expression time is 30 minutes).


Mode Settings

Mode	Suction Level	Frequency (cycles/min)			
Massage	1-5	45		50	
Expression	1-12	20	24	28	32

Pumping with Nouri Duo


1. Wash your hands thoroughly with soap and water and make sure that your breast is clean.

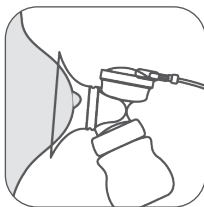2. Place the breast pumps onto your breasts. Make sure that you centre your nipples. If only pumping one breast, make sure to insert the tube plug on the electric pump.

3. Turn on the Nouri Duo by holding the power button. Note pumping will start automatically so make sure the breast is positioned correctly before switching on the pump.

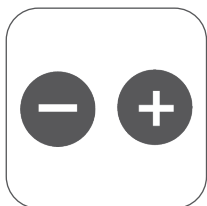
4. When turning on, massage mode will begin. Use the increase and decrease buttons to find a comfortable suction level.

5. To adjust pumping frequency, press and hold mode switch button until the frequency indicator flashes. Then use increase and decrease to select pumping frequency.

6. After 2 minutes, expression mode will automatically begin. Again, select a comfortable suction level and frequency.

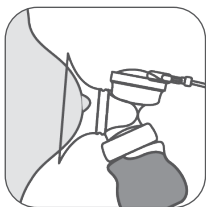7. Stop pumping when the milk stops flowing or when the bottle is full. To turn off, hold the power button.

8. Remove the tube from the bottle assembly.

9. Unscrew the bottle from the shield connector. Use the bottle cap to seal the bottle for storage or the feeding teat for feeding.

Note:

- The pump will run for 30 minutes before automatically turning off. A single expression should not last longer than 20 minutes.
- User can pause the expression by tapping the power button. To begin pumping again, tap the power button again.
- Both breasts must use the same pumping settings.
- Night light has 2 brightness settings. Press the light button to cycle between OFF, low brightness and high brightness.

Advice

- Hot compress your breast before pumping. Pumping after a bath can be useful as the body is warm.
- Whilst you are breastfeeding, pump milk on the other breast, or pump surplus milk after feeding.
- Change the placement position on the breast to stimulate milk secretion
- Practice using Nouri until you find the approach that works best for you. If discomfort occurs during use, please consult a doctor.

Storing Breast Milk

Storage location	Temperature	Max. storage duration
Room	16-29 °C (60-85 °F)	4 hours
Refrigerator	4 °C (39 °F)	4 days
Freezer	<-4 °C (25 °F)	0-6 months

The above only applies for infants one month of older. Please consult a doctor for more details.

Warning:

- Do not refreeze thawed milk.
- Do not mix fresh breast milk with thawed milk.

Note:

- Fresh breast milk should be stored in refrigerator or freezer.
- Be sure to date the milk for storage. Milk should be stored in a refrigerator for no longer than 48 hours and the freezer for no longer than 3 months.
- When freezing, maximum milk volume should be $\frac{3}{4}$ of storage bottle. Milk can expand during freezing.

Preparing Breast Milk

- If the milk is refrigerated, put the milk container in warm water to heat the milk. Shake the container to warm the milk through.
- If the milk is frozen, defrost in the refrigerator 24 hours prior to using. Once defrosted, warm it as refrigerated milk.

Using milk heater:

Turn on milk heater and set it to 40°C. Put milk container into heater. Layers of liquid may form when using frozen milk. This is normal. Please shake bottle well before feeding.

Warning:

- Do not heat thawed milk again.
- Do not use microwave oven to defrost breast milk.
- Do not heat milk with boiling water.
- If hot water is used to heat milk, please try milk temperature before feeding.

Feeding Breast Milk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, teat and other parts before and after each use. If the teat appears cracked or torn, stop using immediately.
- To prevent possible choking hazard, test the strength of teat by pulling on bulb portion of the teat.
- Do not attempt to enlarge the teat hole.
- Infants must not be bottle fed without adult supervision.
- The teat should not be used as a pacifier.
- Clean and disinfect the bottle before first use and after every use.

Charging your Nouri Duo

To charge the Nouri Duo, connect the USB charging cable to the pump and a power socket. The charging indicator and battery level will show on the display.

When fully charged, the battery indicator will stop flashing and show a full battery.

The Nouri Duo can be used during charging if needed.

The battery has a charge time of 3 hours and will last for 90 minutes before needing to be recharged.

When to start using Nouri

1. If you start breastfeeding successfully, we suggest to start using Nouri after regular milk secretion and feeding is formed. (Breast pumps are usually used 2-4 weeks after the baby is born)
2. Nouri Pump should be used in the following circumstances
 - If you cannot start breastfeeding, regular pumping helps establish and sustain milk secretion.
 - When you feed your baby in hospital.
 - When you feel your breast is swelling or painful, you can pump milk between feeding to release discomfort.
 - If your nipple is painful or cracked and therefore unable to feed your baby, you can pump milk until the nipple recovers.
 - If you have to leave your baby for a period of time and therefore not breastfeed regularly, you can pump milk to keep functional milk secretion.

Troubleshooting

Problem	Possible causes	Solution
Pump does not work	<ul style="list-style-type: none">• Empty battery	<ul style="list-style-type: none">• Recharge battery
Low suction force	<ul style="list-style-type: none">• Air leakage at connection positions (Valve, diaphragm, silicone breast shield, tubing)• The output air path of breast pump is blocked, e.g. the suction pipe is pressed or bent, or there are sundries in the pipe• No cause found	<ul style="list-style-type: none">• Check all connecting parts to make sure they are installed properly• Straighten or clean air tubing• Please contact supplier
No suction	<ul style="list-style-type: none">• Valve is not installed or damaged due to deformation• No case found	<ul style="list-style-type: none">• Reinstall or replace valve• Please contact supplier

Warnings

- This product is a precision electronic appliance. Please do not use it during bathing. Please keep it in a cool, dry and ventilated place, avoid direct sunlight, and keep it out of the reach of children.
- Unplug the power adapter after use.
- This product is for personal use only. Do not use it between more than one person to avoid cross infection.
- Do not use a breast pump during pregnancy, because early breastmilk expression will stimulate uterine contraction which may lead to abortion.
- Do not forcibly remove breast shield from breast. Please power off and gently release the seal between the breast and the breast shield, and then remove breast shield from the breast.
- When disassembling or assembling the pump, make sure to assemble each part according to the Installation Guide. When disassembling and cleaning, please be careful not to misplace/lose the small parts.
- Set the suction level according to individual physique to ensure comfortable milk expression.
- Please clean with water and disinfect before use.

Technical Information

Technical Information	
Model Number	K-NRIDUO
Power Input	AC100-240V, 50-60Hz, 0.3A
Power Output	5V-1A
Rated Power	3.6W
Battery Capacity	3.7V/800mAh
Net Weight	367g
Executive Standard	GB 4806.7-2016 GB 4806.11-2016 GB 4806.2-2015

Warranty

This product is warranted to be free from manufacturing defects for one year from the date of purchase. This warranty is void if the product is modified or altered, is subject to misuse or abuse; damaged in transit; lack of responsible care; is dropped; if damage occurs by reason of failure to follow the written instruction booklet enclosed; or if product repairs are carried out without authority from TensCare Ltd.

We will repair, or at our option replace free of charge, any parts necessary to correct material or workmanship, or replace the entire unit and return to you during the period of the warranty. Otherwise, we will quote for any repair which will be carried out on acceptance of our quotation. The benefits conferred by this warranty are in addition to all other rights and remedies in respect of the product, which the consumer has under the Consumer Protection Act 1987.

Our goods come with guarantees that cannot be excluded under the UK consumer Law. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality.

Before you send your unit for service please be sure to read the user manual and follow all the instructions.

Should repair be needed within the warranty period, please return the device with the following details:

1. Name:
2. Address:
3. Telephone:
4. E-mail:
5. Date of Purchase:
6. Proof of Purchase:

Returns should be sent to the following address:

TensCare Ltd
PainAway House,
9 Blenheim Road,
Longmead Business Park,
Epsom, Surrey
KT19 9BE, UK

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TensCare Limited
TensCare Ltd. 9 Blenheim Rd,