nouri pump

User Manual
Manual del usuario
Mode d'emploi
Benutzerhandbuch
Manuale d'uso

TensCare™

Contents

Contents	
Introduction	3
Kit Contents	3
Pump Parts	3
Cleaning and Sterilising	4
Installation Guide	5
Pumping with Nouri	6
Storing Breast Milk	7
Preparing Breast Milk	8
Feeding Breast Milk	8
When to start using Nouri	9
Troubleshooting	9
Materials	9
Warranty	10


Introduction

Nouri Pump is a manual breast pump, designed to provide a comfortable experience whilst you express milk. The pump offers two suction levels and gently stimulates milk flow through the soft massage cushions. Nouri Pump is also compact for on-the-go use.

Kit Contents

- 1 x Manual pump
- 1 x Bottle (150ml)
- 1 x Bottle cap
- 1 x Feeding teat and cap
- 1 x Shield cover
- 2 x Pump valve

Pump Parts

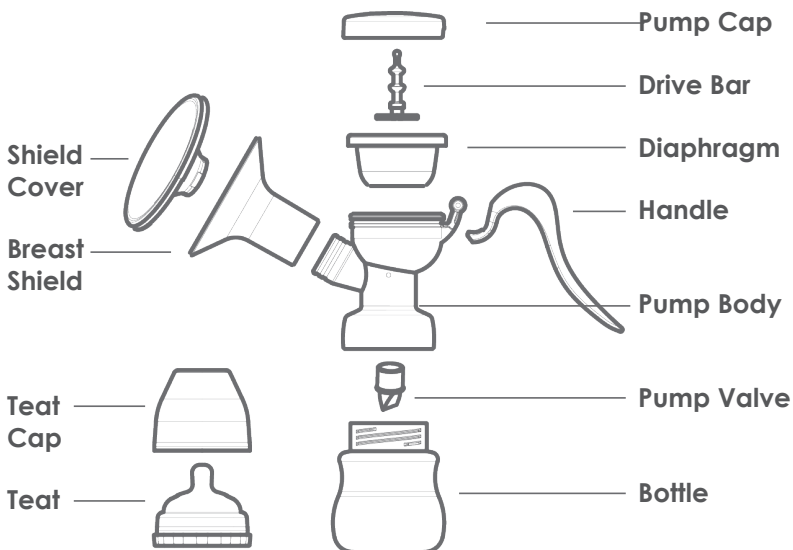Cleaning and Sterilising

Before using for the first time:

Clean and sanitise the parts before using your Nouri Pump for the first time. Use the following cleaning and sanitising instructions.

After each use:

Clean all parts that come in contact with your breast and breast milk:

- Separate all parts, remembering to remove the pump valve from the pump body.
- Rinse the parts in cool water to remove breast milk residue.

Washing in sink:

- Soak all parts in warm soapy water for 5 minutes and then clean with a clean dish-cloth or soft brush.
- Rinse all parts with clear water.
- Allow parts to dry.

Washing in dishwasher:

- Wash all separated parts on the top rack of the dishwasher.
- Allow all parts to dry.

Sanitise daily:

- Wash hands thoroughly.
- Separate all parts that come in contact with your breast and breast milk, remembering to remove the pump valve from the pump body.
- Immerse parts in boiling water for 10 minutes using a household pot.
- Allow water to cool before removing parts from the water.
- Place parts on a clean surface and/or towel and allow to dry.

Caution:

Only use drinking tap water or bottled water for cleaning.

Separate and wash all parts that come in contact with the breast and milk immediately after use to avoid milk residue drying and to prevent bacteria growth.

Do not store wet or damp parts.

During sanitising with boiling water, prevent the parts from touching the side of the pot. This can cause the parts to deform or be damaged.

Nouri is for single person use. To avoid contamination, please do not share the pump.

Note:

Wash the Nouri Pump parts after every use

Sanitise the Nouri Pump parts once a day.

Be careful not to damage parts during cleaning.

Installation Guide

1. Wash your hands thoroughly with soap and water.

2. Insert the drive bar into the diaphragm.

3. Insert the diaphragm into the pump body.

4. Push the pump cap onto the top of the pump body.

5. Insert the pump valve into the bottom of the pump body.

6. Screw the pump body onto the bottle.

7. Attach the breast shield to the pump body.

8. Push the pump handle onto the pump body axle until it snaps into place.


9. Select suction level by attaching the handle to lower or upper section of the drive bar.

Pumping with Nouri

1. Wash your hands thoroughly with soap and water and make sure that your breast is clean.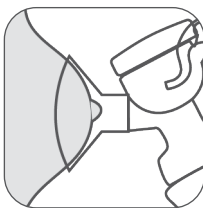

2. Place the breast pump onto your breast. Make sure that you centre your nipple.

3. Softly begin squeezing the handle until you reach a comfortable suction level.

4. Continue to squeeze the handle at 2-3 seconds intervals, imitating the baby's natural suction frequency.

5. Stop pumping when the milk stops flowing or when the bottle is full.

6. Unscrew the bottle from the pump body. Use the bottle cap to seal the bottle for storage or the feeding teat for feeding.

Note:

- It's suggested to wear a breast-feeding bra to facilitate feeding and milk expression.
- Refrigerated milk should be used as soon as possible to avoid spoilage.
- If milk is not being expressed during pumping, try again later.
- Do not overfill the feeding bottle. Bottle should be stored vertical to prevent any leaking.

Advice

- Hot compress your breast before pumping. Pumping after a bath can be useful as the body is warm.
- Whilst you are breastfeeding, pump milk on the other breast, or pump surplus milk after feeding.
- Change the placement position on the breast to stimulate milk secretion.
- Practice using Nouri until you find the approach that works best for you. If discomfort occurs during use, please consult a doctor.

Caution

- Nouri is for single person use. To avoid contamination, please do not share the pump.
- Do not use Nouri during pregnancy. Using breast pumps too early can cause uterine contraction and can lead to abortion.

Please clean and sterilize the product before using.

- Do not use breast pump for over 20 minutes. Using for extended periods of time can cause pain in the breasts.
- Store the pump in a cool and dry place. Keep out of direct sunlight and children's reach.

Storing Breast Milk

Storage location	Temperature	Max. storage duration
Room	16-29 °C (60-85 °F)	4 hours
Refrigerator	4 °C (39 °F)	4 days
Freezer	<-4 °C (25 °F)	0-6 months

The above only applies for infants one month of older. Please consult a doctor for more details.

Warning:

- Do not refreeze thawed milk.
- Do not mix fresh breast milk with thawed milk

Note:

- Fresh breast milk should be stored in refrigerator or freezer.
- Be sure to date the milk for storage.
- When freezing, maximum milk volume should be $\frac{3}{4}$ of storage bottle. Milk can expand during freezing.

Preparing Breast Milk

- If the milk is refrigerated, put the milk container in warm water to heat the milk. Shake the container to warm the milk through.
- If the milk is frozen, defrost in the refrigerator 24 hours prior to using. Once defrosted, warm it as refrigerated milk.

Using milk heater:

Turn on milk heater and set it to 40°C. Put milk container into heater. Layers of liquid may form when using frozen milk. This is normal. Please shake bottle well before feeding.

Warning:

- Do not heat thawed milk again.
- Do not use microwave oven to defrost breast milk.
- Do not heat milk with boiling water.
- If hot water is used to heat milk, please try milk temperature before feeding.

Feeding Breast Milk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, teat and other parts before and after each use. If the teat appears cracked or torn, stop using immediately.
- To prevent possible choking hazard, test the strength of teat by pulling on bulb portion of the teat.
- Do not attempt to enlarge the teat hole.
- Infants must not be bottle fed without adult supervision.
- The teat should not be used as a pacifier.
- Clean and disinfect the bottle before first use and after every use.

When to start using Nouri

1. If you start breastfeeding successfully, we suggest to start using Nouri after regular milk secretion and feeding is formed. (Breast pumps are usually used 2-4 weeks after the baby is born).
2. Nouri Pump should be used in the following circumstances:
 - If you cannot start breastfeeding, regular pumping helps establish and sustain milk secretion.
 - When you feed your baby in hospital.
 - When you feel your breast is swelling or painful, you can pump milk between feeding to release discomfort.
 - If your nipple is painful or cracked and therefore unable to feed your baby, you can pump milk until the nipple recovers.
 - If you have to leave your baby for a period of time and therefore not breastfeed regularly, you can pump milk to keep functional milk secretion.

Troubleshooting

Problem	Possible causes	Solution
Pump does not work	Low suction force or no suction force	<ol style="list-style-type: none">1. Make sure shield cushion closely touches breast.2. Check if shield cushion is firmly installed on breast shield.3. Make sure that diaphragm outer ring connects with breast shield properly.4. Check if handle is fully inserted in diaphragm.5. Make sure the valve is installed accurately.

Materials

Part	Material	Maximum Temperature/°C
Breast Shield Connector	PP	110
Handle	PP	110
Cap	PP	110

Part	Material	Maximum Temperature/°C
Feeding Bottle	PP	110
Drive Bar	PP	110
Diaphragm	Silica Gel	120
Valve	Silica Gel	120
Teat	Silica Gel	120

Warranty

This product is warranted to be free from manufacturing defects for one year from the date of purchase. This warranty is void if the product is modified or altered, is subject to misuse or abuse; damaged in transit; lack of responsible care; is dropped; if damage occurs by reason of failure to follow the written instruction booklet enclosed; or if product repairs are carried out without authority from TensCare Ltd.

We will repair, or at our option replace free of charge, any parts necessary to correct material or workmanship, or replace the entire unit and return to you during the period of the warranty. Otherwise, we will quote for any repair which will be carried out on acceptance of our quotation. The benefits conferred by this warranty are in addition to all other rights and remedies in respect of the product, which the consumer has under the Consumer Protection Act 1987.

Our goods come with guarantees that cannot be excluded under the UK consumer Law. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality.

Before you send your unit for service please be sure to read the user manual and follow all the instructions.

Should repair be needed within the warranty period, please return the device with the following details:

1. Name:
2. Address:
3. Telephone:
4. E-mail:
5. Date of Purchase:
6. Proof of Purchase:

Returns should be sent to the following address:

TensCare Ltd
PainAway House,
9 Blenheim Road,
Longmead Business Park,
Epsom, Surrey
KT19 9BE, UK

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TensCare Ltd
 PainAway House,
 9 Blenheim Road,
 Longmead Business Park,
 Epsom, Surrey
 KT19 9BE, UK

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Notes

Follow us

TensCare aim to give you the best possible product and service. We listen to your suggestions and are constantly trying to improve our products. We also want to learn about the way our products are used, and the benefits they give. If you have anything you would like to share with us, please get in touch.

TensCare Ltd

@TensCareLtd

TensCare Limited

@tenscareLtd

nouri

TensCare Ltd. 9 Blenheim Rd,
Epsom, Surrey, KT19 9BE, U.K.
+44 (0) 1372 723 434

I-NRIPUMP-ML [UK,ES,FR,DE,IT] Rev 1.0 08/21

TensCare[™]