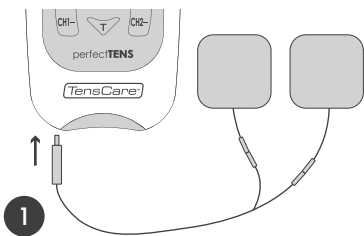
perfect**TENS**


Instructions for Use
Please read carefully

TensCare™


Quickstart Guide


1 Connect the device to the electrodes.

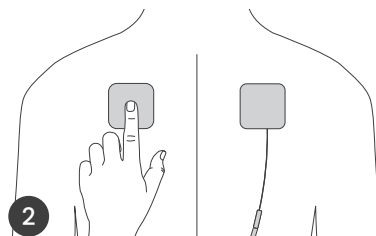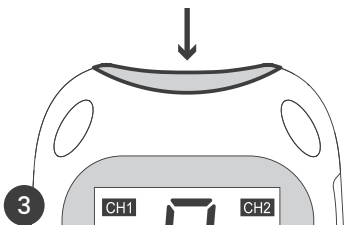
2 Place the electrodes on the area to be treated.

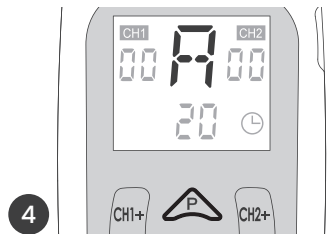
3 Press and hold the power button to turn the device on.

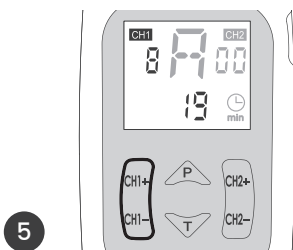4 Select a programme by pressing the P button.

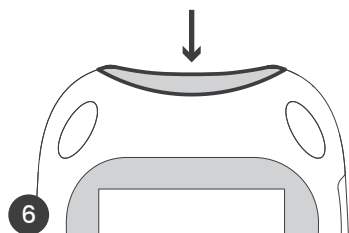5 Start the session by increasing the intensity.

6 Press and hold the power button to turn the device off.

Contents

1. Device Description	
1.1. Introduction	3
1.2. Intended purpose	3
1.3. Kit contents	3
1.4. Treatment options	3
1.5. Controls and display	5
2. How to Use	
2.1. Device setup	5
2.2. Electrode placement	6
2.3. Completing a treatment	8
2.4. After a treatment	11
2.5. Treatment guidance summary	12
3. Troubleshooting	12
4. Device Care	13
4.1. Cleaning	13
4.2. Storage	13
4.3. Battery replacement	13
4.4. Electrode and lead wire care	14
4.5. Replacement parts/accessories	15
5. Technical Specification	15

1. Device Description

1.1. Introduction

The Perfect TENS is a clinically proven, dual-channel electrotherapy stimulator that uses Transcutaneous Electrical Nerve Stimulation (TENS) to provide pain management and relief.

The device delivers small electrical signals to the body via body-worn, adhesive electrodes. These signals interact with the muscles and/or nerves. When used as intended, the Perfect TENS can provide the following clinical benefits:

- Reduction in pain
- Reduced pain medication reliance

1.2. Intended purpose

Perfect TENS is a medical device designed to be used in the home healthcare environment to provide symptomatic relief and management of acute or chronic localised pain and muscle pain and is suitable for use by all who can control the device and understand the instructions.

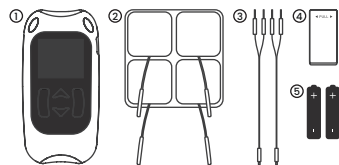
Do not use the device for any purpose other than this intended use.

 **Warning:** Not suitable for use in children without medical supervision.

1.3. Kit contents

- 1 x Perfect TENS device
- 4 x 50x50mm electrodes (E-CM5050)

- 2 x Lead wires (L-CPT)
- 1 x Detachable belt clip
- 2 x AA batteries
- 1 x Storage pouch
- 1 x Instructions for Use

1.4. Treatment options

TENS is a clinically proven and safe treatment option for providing pain relief. TENS works in one of two ways:

Pain Gate

Stimulates sensory nerves to block pain signals from being received by the brain. This is achieved through high-frequency settings ranging from 80 to 120Hz. Pain Gate stim provides pain relief whilst being used.

Endorphin Release

Low frequency TENS (typically 2-10Hz) promotes the release of endorphins, the body's natural pain killer. Endorphin Release builds up over 40 minutes and will last several hours after stimulation is ended.

The TENS programmes have either constant or burst output.

Constant output delivers a continuous stimulation, providing a constant, consistent sensation.

Burst output follows an On/Off pattern, giving a pulsing or bursting sensation. This combines the benefits of high-frequency and low-frequency TENS.

Select the desired treatment from the programmes below.

Prog.	Type	Output	Type of relief	Description
A	Preset	Constant	Pain gate	Most gentle preset TENS programme. Recommended for your first use to get accustomed to the tingling sensation of TENS.
B	Preset	Constant	Endorphin release	Low frequency programme ideal for chronic pains such as back pain and arthritis.
C	Preset	Burst	Pain gate & endorphin release	High frequency programme in low frequency bursts. Effective for deep muscular pain.
D	Preset	Constant	Endorphin release	Low frequency programme for promoting endorphin release. Provides a soothing/relaxing effect.
E	Preset	Constant	Pain gate	A stronger version of programme A that provides a stronger TENS sensation.
F	Preset	Pulse width modulated	Pain gate	This programme constantly changes pulse width to prevent accommodation occurring. Provides a slow relaxing massage sensation.
G	Preset	Frequency modulated	Pain gate	This programme constantly changes frequency to prevent accommodation occurring. Ideal for treating chronic pain over a long period of time.
H	Preset	HAN	Pain gate & endorphin release	Alternates between low and high frequency cycling between 3 seconds of a tapping sensation followed by 3 seconds of tingling sensation.
M	Manual	Constant / Burst	Pain gate & endorphin release	Manual TENS programme for using bespoke treatment settings.

i **Note:** TENS programmes may produce different results for different users. It is recommended to experiment with the available programmes and electrode positioning to optimise your TENS therapy for maximum pain relief.

1.5. Controls and display

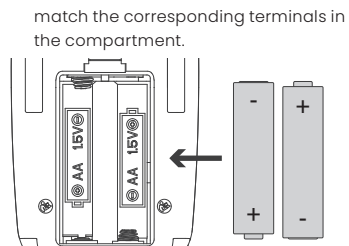
2. How to Use


2.1. Device setup

Install the batteries

1. Remove the battery cover on the rear of the device.
2. Insert the batteries, ensuring that the positive (+) and negative (-) ends

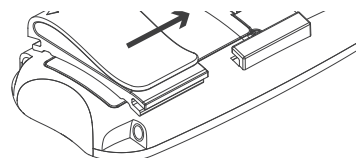
3. Replace the battery cover.
4. To use the belt clip, slide the belt clip into the rear channels of the device.

Connect the leads

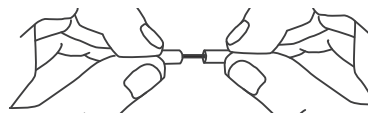1. Insert the white end of the lead into the channel socket on the rear of the device.
2. Push the metal pins of the lead wires into the sockets on the electrodes.


Position the Electrodes

1. Before applying the electrodes, make sure your skin is clean and dry
2. Select the electrode position based on the selected treatment mode and area of the body being treated. See guidance in the following pages.

The electrodes must always be used in pairs (two electrodes per channel) to ensure the signal can flow in a circuit. The positive (+) electrode should be connected to the red lead wire, and the negative (-) electrode should be connected to the black lead wire.

3. Remove the electrodes from the plastic liner.

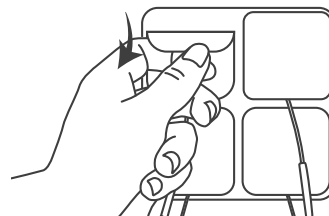4. Apply the electrodes to the body.

i **Note:** always ensure the device is turned off or the intensity is zero before applying the electrodes.

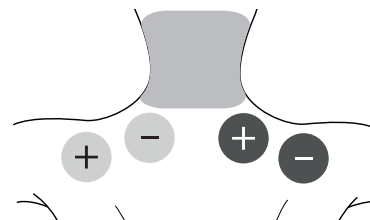
2.2. Electrode placement

The easiest way is to apply the electrodes around/near the source of the pain.


TENS activates the nerves best if it travels along the nerve rather than across it. So, place one electrode further from the spine than the source of the pain, and one closer.

If the pain is in, or close to, your spine, place one electrode either side of the spine.


Neck pain


Upper back pain

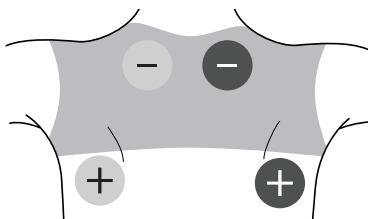Lower back pain

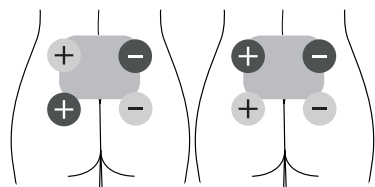Ankle and foot pain

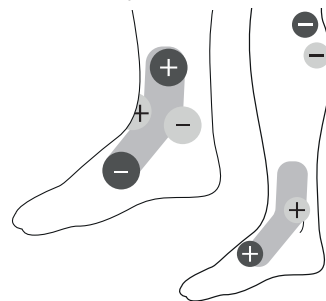2.3. Completing a treatment

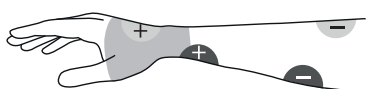
Turning on / off

1. Turn the device on/off by pressing and holding the power button located on the top for 3 to 5 seconds.


i Note: The device will automatically turn off if the intensity setting is left at zero for more than 5 minutes.


Wrist pain


Knee pain

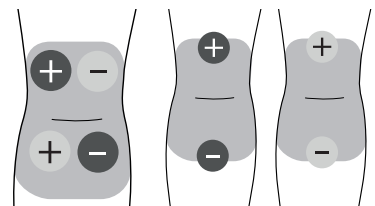
Sciatica

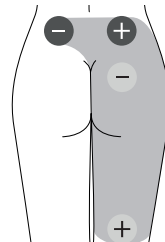Shoulder pain

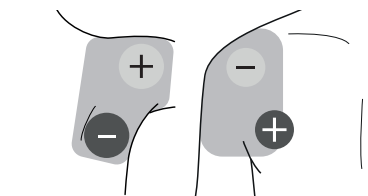Elbow pain

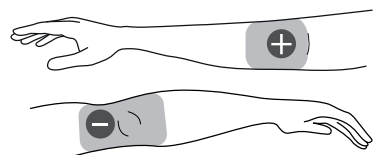Programme selection

1. Select the desired programme by pressing the P button. (See section 1.4 Treatment options).

Each time you change the programme the intensity level will revert to zero.

When turning on, the device will remember the last programme used.

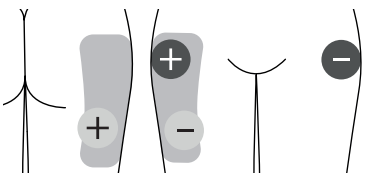
Treatment time selection

The default treatment time for each programme is 20 minutes.


1. To change the treatment time, press the T button. The 'min' icon should flash on the display.


Hip pain

Period pain

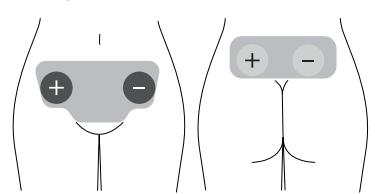Trigeminal Neuralgia

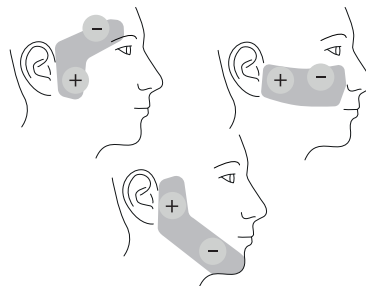- Press CH1+- to change the treatment time from either 10, 20, 30, 45, 60, 90 minutes or continuous.
- Press T again to confirm the treatment time.

TENS can be used for as long as it is necessary. Continuous treatment is fine, but the electrodes should be repositioned at least every 12 hours to allow the skin to be exposed to the air.

Low frequency TENS works better when the intensity is high enough to cause small muscle contractions. Best results are

Prog	Frequency (Hz)	Pulse Width (μ s)	Treatment time (min)	Output
M	2-150	50-250	Constant (C), 10,20,30,40,60	Constant (C) or Burst (B)

Parameter	Description	Icon
Frequency	The number of pulses per second.	Hz
Pulse Width	The duration of an individual pulse.	μ s
Treatment timer	The duration of the treatment	 Timer icon
Constant / Burst Output	The type of stimulation for TENS	C/B

- To adjust the settings, press the T button.
- Press the T button again to cycle through the various adjustable settings.
- Adjust the parameters by pressing CH1+.

The Hz symbol will flash on the screen, indicating that the frequency setting is ready to be adjusted.

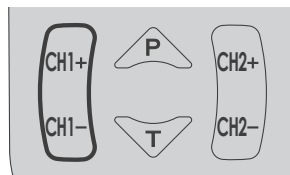
achieved with a session duration between 20 and 40 minutes. Longer use may cause muscle ache.

If unsure, start with a 20-minute treatment session and increase/decrease as necessary.

Manual programme setting selection

Programme M is a manual programme that allows for bespoke selection of treatment parameters. The adjustable parameters and corresponding screen icons can be seen below:

- After selecting all the parameters and setting the treatment time, press the T button once more to confirm and save your settings.

Starting a session

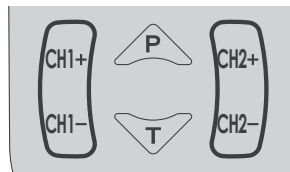
With the device set-up and the preferred programme and treatment time selected, start the treatment session by increasing the intensity using CH1+ or CH2+ (depending on which channel is connected to your body).

Intensity selection

With the treatment started, select the stimulation intensity by using the CH+- buttons.

CH1+ CH1- control the intensity of the left-hand lead/electrodes.

CH2+ CH2- control the intensity of the right-hand lead/electrodes.

Perfect TENS has 60 levels of intensity. Each press of the + and - buttons will adjust

the intensity by 1mA. The intensity can be increased or decreased quickly by pressing and holding the +- buttons.

Intensity cannot be increased if the device has not been correctly connected to the body (i.e. a closed circuit is made). If the perfect TENS detects a disconnection in either channel it will automatically return the strength to zero in that channel and flash '00'.

To increase intensity, ensure the electrodes are correctly connected to the body and the device.

The stimulation sensation should be strong but comfortable. Further increases during use may be necessary if your body becomes used to the sensation. At low intensity levels, it is common for no sensation to be felt. Increase the intensity slowly to find a level that is suitable for you.

Intensity may need to be increased mid-way through treatment as your body gets used to the sensation. If the sensation becomes uncomfortable, reduce the intensity.

When the yellow LEDs in the channel output sockets light, this indicates the stimulation intensity is high enough to be harmful if not used correctly. Please use with caution.

Keypad lock

If you do not press any keys for 30 seconds, the keypad will lock. This is to avoid accidental changes in setting. To unlock, press either CH1- or CH2- keys.

Low battery

When the battery is low, the battery icons will display on the screen. The device will automatically turn off after 2 minutes.

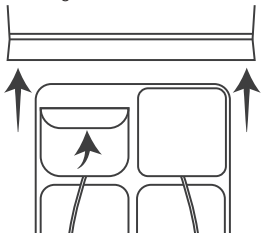
Replace with new AA batteries.

2.4. After a treatment

Disconnecting the device

Once the treatment timer reaches zero, the treatment session will end, and the device turns off automatically.

1. Remove the electrodes from the skin by holding the pad and gently pulling. Do not pull on the lead wires.
2. Replace the electrodes on the plastic liner.
3. Disconnect the leads from the electrodes and device.
4. Place the electrodes in the resealable plastic bag.

5. Return the device, leads and electrodes to the storage pouch.

Frequency of use

TENS can be used as frequently as necessary and is approved for daily use.

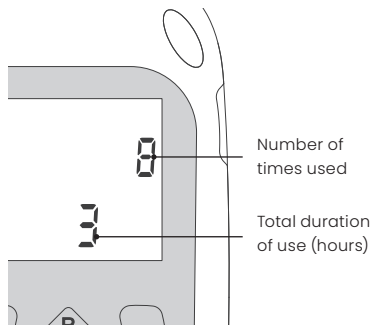
Usage memory

The Perfect TENS tracks the number and duration of use.

To view the usage memory, press T and CH2- buttons for 3-5 seconds. The display will show the number of times used and duration in hours used.

Press the same buttons again to return to normal controls.

To reset memory to zero, hold down the T and CH1- buttons together for 3 to 5 seconds.

2.5. Treatment guidance summary

	Treatment time	Intensity	Frequency of use
TENS	High-frequency TENS – as long as necessary to provide pain relief. Low-frequency TENS – 20-40 minutes.	Intensity level which provides strong but comfortable sensation.	TENS can be used as frequently as necessary and is approved for daily use.

3. Troubleshooting

If the device is not working correctly, use the following troubleshooting guide.

Problem	Possible causes	Solution
Display will not turn on	Flat batteries.	Replace batteries.
	Batteries inserted incorrectly.	Remove plastic wrap. Check the positive and negative ends are positioned correctly.
	Damaged springs in battery compartment.	Contact supplier.
Controls will not work	The keypad is locked.	If the key icon is shown, press either CH1- or CH2- to unlock the device.
Low battery display	Low batteries.	Replace batteries.
No sensation	Intensity is not strong enough.	Increase the intensity. Most users will feel something at a setting below 20 mA.
Output will not increase above zero	There is a connection error leading to an open circuit.	Ensure machine is attached to your body correctly.
	Electrodes are not connected to body.	Try using the second lead wire.
	Leads are faulty/damaged.	Purchase replacement if necessary.

Sudden change in sensation	If you stop the stimulation and continue a few minutes later, the signal will feel quite a lot stronger.	Always return the intensity to zero after disconnecting the leads or electrode pads.
----------------------------	--	--

If you are unable to resolve the issue, please contact our customer service team:

Email: customer.service@tenscare.co.uk

Phone: +44 1372 723434

i Note: The Perfect TENS has no essential performance characteristics. Device failure will not cause unacceptable clinical risk for the user.

4. Device Care

4.1. Cleaning

- Clean the device and lead wires at least once a week by wiping with a damp cloth and solution of mild soap and water. Wipe dry.
- Do not immerse the device in water.
- Do not use any other cleaning solution than soap and water.

4.2. Storage

- Between uses it is important to store the electrodes in their resealable plastic bag with the bag sealed. This will help extend the life of the electrodes.
- Keep the device and all accessories in the storage pouch between uses.

Storage life

- Storage life of an unopened pack of self-adhesive electrode pads is 4 years. This may be affected by very high temperatures or very low humidity.

- The device has no fixed shelf life.
- Unused batteries have a nominal shelf life of 3 years but will usually last longer than this.

4.3. Battery replacement

- When the batteries are low and the battery icon is on the display, replace the batteries.
- Batteries should last at least 15 hours at 50 mA, 300 µs, 50 Hz.
- The unit will work with rechargeable batteries, but the display may appear dim.

Battery Warnings

- Remove batteries from the device if it is unlikely to be used for a long period. Some types of batteries may leak corrosive fluid.
- Do not pierce, open, disassemble, or use in a humid and/or corrosive environment.
- Do not expose to temperatures over 60°C (140°F).

- Do not put, store or leave near sources of heat, in direct strong sunlight, in a high temperature location, in a pressurized container or in a microwave oven.
- Do not immerse in water or get wet.
- Do not short-circuit.
- Do not connect the device unless the battery cover is in place.
- If battery leakage occurs and comes in contact with the skin or eyes, wash thoroughly with lots of water and immediately seek medical attention.
- Keep batteries out of the reach of children to prevent them from swallowing them by mistake. If swallowed by child, contact doctor immediately.
- Never attempt to recharge an alkaline battery. Risk of explosion.
- Do not mix old, new or different types of batteries as this may lead to battery leakage or low battery indication.
- Disposal: Always dispose of batteries and device responsibly according to local government guidelines. Do not throw batteries onto a fire. Risk of explosion.
- To maximise the life of the electrodes, always ensure the electrodes are replaced on to their plastic liner and stored and sealed in the plastic bag. When the electrodes dry, this may negatively impact the performance of the treatment.
- The adhesive is the electrode is a peelable hydrogel (water based). To restore the tackiness of the electrode, add a tiny drop of water and spread it around. Avoid adding too much, as it will make the electrode too soft. If this occurs, place the electrodes sticky side up in a refrigerator for a few hours to regain adhesiveness.
- Replace the electrodes when they lose their stickiness. Poor connection may cause discomfort and skin irritation.
- To ensure safety and effectiveness we recommend using TensCare electrodes only.
- The life of the lead wires depends greatly on use. Always handle the leads with care. We recommend replacing the lead wires regularly (about every 6 months).

4.4. Electrode and lead wire care

- The electrodes are reusable but for single patient use. The electrodes should provide up to 20 uses. This may vary depending on skin condition, humidity and storage between uses.

4.5. Replacement parts/ accessories

The following replacement parts are compatible with the Perfect TENS and may be ordered from TensCare at www.tenscare.co.uk or +44(0) 1372 723434.

E-CM5050	50x50mm electrodes (pack of 4)
E-CM50100	50x100mm electrodes (pack of 4)
E-CR5050	50x50mm Cool Relief infused electrodes (pack of 4)

E-CR50100	50x100mm Cool Relief infused electrodes (pack of 4)
E-TRBG	50x50mm Blue Gel hypoallergenic electrodes (pack of 4)
E-CMR60130	Breast electrodes (pack of 4)
L-CPT	Lead wire
B-AA	AA batteries (pack of 2)

5. Technical Specification

Waveform	Symmetrical Bi-Phasic
Amplitude (over 500 Ohm load)	80 mA zero to peak +ve in 60 steps +/- 10%
Frequency range	2-150Hz
Pulse width range	50-250us
Output plug	Fully shielded: touch proof
Channels	Dual channel
Power supply	2 x AA alkaline (two AA batteries)
Weight	75g (without batteries)
Dimensions	120 x 60 x 20 mm


Safety Classification Internal power source. Designed for continuous use. No special moisture protection.

Service life	>5 years
Software version	PTensA_M6832_R6
Applied part	Skin surface electrodes
Applied part classification	Type BF
IP rating	IP22

Environmental Specifications

Operating conditions	Temperature range: 5 to 40°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa
Storage conditions	Temperature range: -25 to +60°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa

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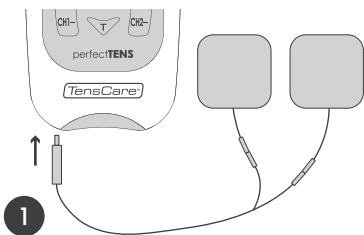Thank you for using Perfect TENS...

At TensCare, we strive to provide the best possible products and services. We value your feedback and continuously work to enhance our offerings. Understanding how our products benefit you is important to us. If you have any insights or suggestions, please get in touch.

Contact us

TensCare Ltd.
9 Blenheim Road,
Epsom, Surrey,
KT19 9BE, U.K.

+44 (0) 1372 723434

www.tenscare.co.uk

customer.service@tenscare.co.uk

Follow us

TensCare Ltd

@tenscareltd

@TensCareLtd

TensCare Limited

EU Declaration of Conformity

Tenscare Ltd herewith declare that the product meets the provisions of REGULATION (EU) 2017/745 which apply to it. The medical device has been assigned to class IIa according to Annex IV of the REGULATION (EU) 2017/745. The product concerned has been designed and manufactured under a quality management system according to Annex IX, of REGULATION (EU) 2017/745. This EU declaration of conformity is issued under the sole responsibility of the manufacturer. No "Common Specification" is applicable.

www.tenscare.co.uk

TensCare Ltd.

9 Blenheim Road, Epsom,
Surrey, KT19 9BE, U.K.

2797

EU DoC

Complies with Medical Device
Regulation (EU) 2017/745.

Advena Ltd.

Tower Business Centre,
2nd Flr, Tower Street, Swatar,
BKR 4013, Malta

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