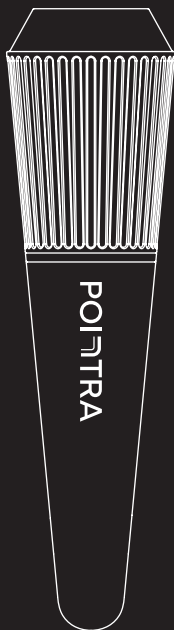
POINTRA

Point. Treat. Move.

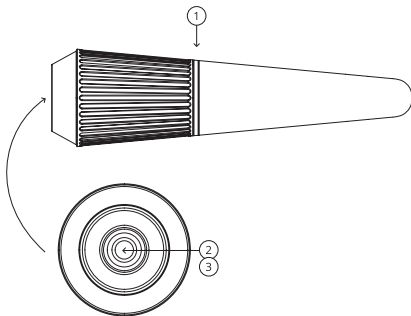
Instructions for use

Please read carefully

TensCare™

Pointra Quick Start Guide

1. Charging Port – Access the charging port by unscrewing the silicone head.
2. Multi-Function Button – Power On/Off & Mode Selection.
3. Battery Indicator Light – Shows battery status and charging progress.

Technical Specification

Maximum Speed: 8000 rpm

Power Supply: 5V / 0.5A

Modes: 10 unique FVT (Focal Vibration Therapy) patterns

Operating Time: Up to 3 hours

Charging Time: Approximately 1 hour

Battery: 320mAh

Max Output: 1 W (max operating power)

Rated voltage: 3.7V

Weight: Approximately 116g

Size: 1.5cm x 3.7cm x 14cm

Turning On/Off

Press the power button to turn Pointra on — it will start in FVT mode 1.

Press the power button again to cycle through the 10 FVT modes.

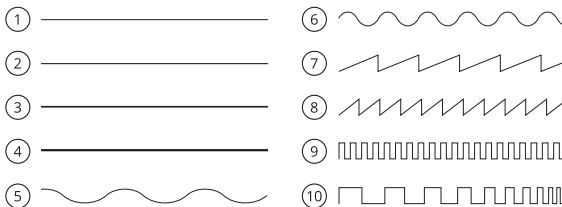
To switch off, press and hold the power button for 1 second.

Pointra FVT Modes

Pointra cycles through 10 FVT modes — designed to target deep tissues, lift fascia layers and stimulate circulation.

Repeatedly press the power button to explore all modes. After the 10th mode, Pointra will return to FVT mode 1.

 **Warning:** Do not exceed 10 minutes of continuous use per session.

Charging Your Device

Charge Pointra fully before your first use to protect battery life.

When the battery is below 20%, the indicator light will flash.

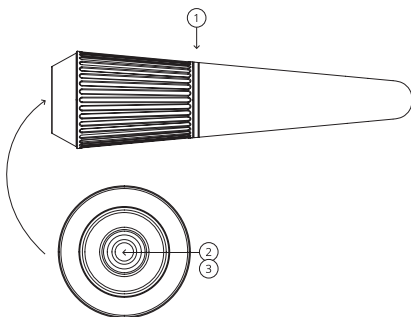
Access the charging port by unscrewing the silicone head.

Use the included charging cable to connect to a USB power adapter.

The light will flash during charging and turn solid when fully charged.

Disconnect the charger after the battery is full.

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SUZHOU PINOTEC CO., LTD. No.38, Jingyang Road,