# i sock

---

FOR FOOT PAIN  
AND ARTHRITIS

**Instructions For Use**  
Please read carefully

Conductive Garment for use with TensCare TENS & EMS units.

- For single patient use only.
- Two socks, supplied with adapter lead.
- Supplied with one pack of two reference electrode pads (50 x 90 mm electrodes).

TensCare have altered the specification of the iSock to improve performance:

- We have changed the fabric to a higher silver content to give improved contact.
- We now supply two socks for use with each foot.
- We now supply self-adhesive reference electrodes, to give a more reliable current distribution.

### **Instructions for Use**

1. Ensure that the stimulator is turned OFF.
2. Insert one pin connector (from one lead pair) into the connector attached to the iSock, and the other (from the same lead pair) into the connector on the electrode pad.
3. The socks should be worn slightly damp to improve current flow.
4. Apply the garment
5. Place the electrode pad on the same limb as the iSock.
6. If necessary, hand wash the iSock and dry flat at room temperature.

### **Caution**

If being used for pain management for Diabetic Neuropathy or other conditions where there is reduced or no sensory sensation then there is a small risk that reduced or uneven electrical contact may cause irritated or broken skin. Care needs to be taken to monitor the skin condition under the garment at frequent intervals.

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**TensCare Ltd.**
**Advena Ltd.**