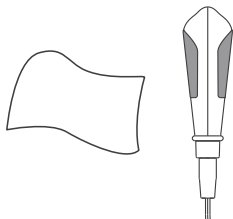
SurePRO

Instructions for Use
Please read carefully


TensCare™

Quickstart Guide


1

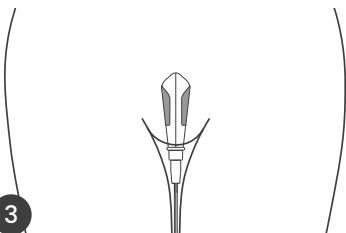Clean the probe with a wipe or with a cloth and some warm soapy water


3

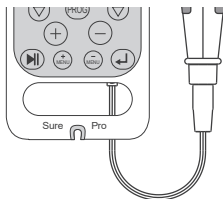Insert the probe into the vagina with the metal plates facing outwards.


2

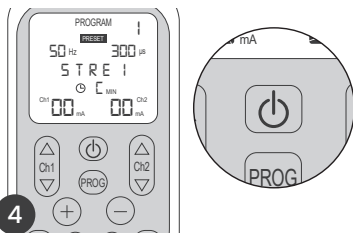Connect the probe to the device.


4

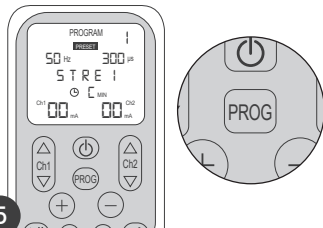Press and hold the power button to turn the device on.

5

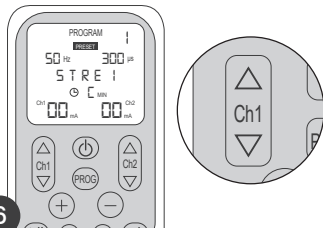Select a programme by pressing the programme selection button.

6

Start the session by increasing the intensity.

Contents

1.	Device Description	3
1.1.	Introduction	3
1.2.	Intended purpose	3
1.3.	Kit contents	4
1.4.	Treatment programmes	4
1.5.	Controls and display	13
2.	How to Use	14
2.1.	Device setup	14
2.2.	Starting a session	18
2.3.	Completing a treatment	18
2.4.	Alternative electrode treatment	19
2.5.	Alternative anal probe treatment	19
2.6.	Kegel exercises	20
3.	Troubleshooting	21
4.	Device Care	23
4.1.	Cleaning	23
4.2.	Storage	23
4.3.	Charging Sure Pro	23
4.4.	Probe and lead wire care	25
4.5.	Electrode care	25
4.6.	Replacement parts	25
5.	Technical Specification	26

1. Device Description

1.1. Introduction

Sure Pro is a clinically proven electrical muscle stimulator designed to treat urinary and faecal incontinence and improve pelvic floor health. It uses gentle electrical signals delivered through a vaginal or anal probe to stimulate and contract the pelvic floor muscles, promoting muscle strength and bladder control to help reduce or eliminate leakage. Pelvic floor stimulation may also support sexual health by toning the muscles involved in intimacy.

The **Sure Pro** is a versatile unit suitable for home use, featuring two independent stimulation channels and compatibility with vaginal or anal electrodes, as well as self-adhesive pads for urge incontinence and pain relief. It offers 12 preset programmes and 3 manual programmes, including specialised settings for transcutaneous tibial nerve stimulation for urge incontinence. Manual programmes can be customised by healthcare professionals to meet individual needs.

Sure Pro can provide relief from:

- Urinary and faecal incontinence: stress, urge, mixed types, and post-prostatectomy urinary incontinence in men.
- Chronic pelvic pain: conditions such as vulvodynia, symphysis pubis dysfunction, or interstitial cystitis.
- Primary dysmenorrhoea (period pain).

When used as intended, **Sure Pro** may deliver the following clinical benefits:

- Reduced frequency of stress and urge incontinence episodes
- Improved pelvic floor muscle strength and bladder control
- Relief from pelvic pain and discomfort

1.2. Intended purpose

Sure Pro is a medical device designed for use in the home healthcare environment to treat symptoms of urinary and/or faecal incontinence, including stress, urge, and mixed types. It may also help relieve symptoms of primary menstrual pain (dysmenorrhoea) and chronic pelvic pain, as well as secondary menstrual pain caused by conditions such as endometriosis, fibroids, or pelvic inflammatory disease.

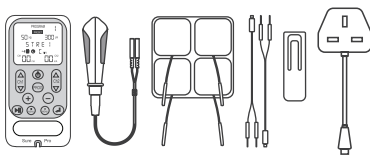
Sure Pro is suitable for use by adults (18 years+) who can control the device and understand the instructions.

Do not use the device for any other purpose.

1.3. Kit contents

- 1 x **Sure Pro** device
- 1 x Liberty Loop vaginal probe (X-VP)
- 4 x 50x50mm electrode pads (E-CM5050)
- 2 x Lead wires (L-ST2)
- 1 x Detachable belt clip
- 1 x Charging adapter
- 1 x Storage pouch
- 1 x Instructions for Use

1.4. Treatment programmes

Each programme has its own combination of Frequency and Pulse Width settings which allow for different sensations through the probe or optional electrode pads and help treating the different types of incontinence.

Frequency (measured in Hz - pulses per second)

Low frequencies (1-10 Hz) have a purifying and relaxing effect through individual contractions.

Medium frequencies (20-50 Hz) can put a high level of strain on the muscle, thus promoting the muscular structure

Endorphin release (programme PAIN only): A low frequency of 4 or 10 Hz allows for the release of endorphins, the body's natural morphine-like substances.

Pulse Width (measured in μ s - millionths of a second)

The **Sure Pro** unit has pulse widths of 50 to 500 μ s. Generally speaking, the higher

the pulse width, the more "aggressive" the stimulation feels,

if the pulse width is set high enough, it will usually elicit a muscle contraction, which is required for an effective toning of the pelvic floor muscles.

The **Sure Pro** has 12 preset programmes, with programmes for each type of incontinence (**STRESS, URGE and MIXED**), as well as other programmes for conditions including loss of sensation and pelvic pain .

1) STRE 1 - Stress Incontinence

The **STRESS** incontinence programme strengthens the muscles of the pelvic floor using gentle stimulation. Once muscular strength has been improved these muscles are better able to resist urinary leakage caused by external pressure being applied to the bladder such as with a cough, sneeze or physical exertion. The stimulation causes the muscles to contract and work thereby building their strength.

Successful treatment requires stimulation once a day for one to three months. Improvement starts becoming apparent after about four weeks. It helps to keep a record of leakage problems so that you have an objective measure of your progress.

The sensation is like a strong drawing in of the muscles of the vagina, pulling up the pelvic floor. Your natural reaction will be to pull your muscles in and up, and this exercises and strengthens them.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

2) STRE 2 – Stress Incontinence, aftercare

Having restored your pelvic floor muscles to an excellent condition with **STRE 1**, you will want to keep them toned and strong.

Regular use of this programme, about twice a week, will ensure that your muscles remains fit and toned.

May also be used as an alternative treatment for **STRE 1**.

The sensation is a mixture of a strong drawing in of the muscles and then releasing. The programme repeats this sensation.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

A strong and fit pelvic floor muscle may increase sexual health and enjoyment.

3) URGE – Urge Incontinence

The **URGE** incontinence programme reduces the involuntary contractions of the bladder (detrusor) muscle. This prevents the unwanted and unexpected emptying of the bladder.

Successful treatment will require daily stimulation and improvements can be seen in as little as two weeks.

The sensation is of a longer and softer pulling in of the pelvic floor than the **STRESS** programme.

It is important to feel this sensation throughout the programme, so you may need to increase the intensity during the treatment session. However, it is not necessary to contract the muscles.

4) MIXED – Mixed Incontinence

This programme is ideal if you suffer both stress and urge incontinence. This programme is a combination of 15 minutes of the **URGE** programme, followed by 15 minutes of the **STRE 1** programme.

During the first part, the sensation in the pelvic floor is soft, like a vibration. It is important to feel this sensation throughout the programme; you may need to increase the intensity during the treatment session. However, it is not necessary to contract the muscles.

In the second part, the sensation is like a strong drawing in of the muscles of the vagina, pulling up the pelvic floor.

Your natural reaction will be to pull your muscles in and up, and this exercises and strengthens them.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

5) URGE2 – Urge Incontinence, aftercare

Similar to the programme 3, but the sensation is continuous (there is no work/rest period), which you may find more comfortable.

The sensation in the pelvic floor is soft, like a vibration. It is important to feel this sensation throughout the programme, so you may need to increase the intensity during the treatment session. However, it is not necessary to contract the muscles.

6) SENS – Lack of Sensation

After surgery or childbirth, you may find that you have good muscle control, but experience a lack of sensitivity. This can be due to nerve damage and is a problem that may be helped with the programme.

The sensation is like a strong drawing in of the muscles of the vagina, pulling up of the pelvic floor. Your natural reaction will be to pull in and up your muscles, thereby exercising and strengthening them.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

7) PFW A – Pelvic Floor Workout

A multiphased programme for Mixed incontinence. The smaller steps in frequency are believed to be less likely to aggravate the Urge component.

The sensation is like a strong drawing in of the muscles of the vagina, pulling up of the pelvic floor. Your natural reaction will be to pull in and up your muscles, thereby exercising and strengthening them.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

8) PFW B – Pelvic Floor Workout, aftercare

Having restored your pelvic floor muscles to an excellent condition with.

9) ENDUR – Endurance

Choose this programme if you feel like you can contract your pelvic floor muscles quite easily, but can't keep them contracted for very long. **ENDUR** programme will help to build up muscle strength and improve muscle endurance.

The sensation is like a strong drawing in of the muscles of the vagina, pulling up of the pelvic floor. Your natural reaction will be to pull in and up your muscles, thereby exercising and strengthening them.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

10) PAIN – Pelvic pain

The **PAIN** programme helps treat pain in the pelvic area. It is particularly useful for treating vulvodynia, a condition that can cause burning, stinging, irritation and rawness in the female genital area.

The sensation in the pelvic floor is soft, like a vibration. It is important to feel this sensation throughout the programme; you may need to increase the intensity during the treatment session. However, it is not necessary to contract the muscles.

PFW A, you will want to keep them toned and strong with this programme which has a longer Work time (contraction time).

The sensation is the same as **PFW A**.

It is necessary that the pelvic floor muscles contract. Increase the intensity level as high as you can tolerate, then reduce by 1 mA.

11) TIBN 1 – Transcutaneous tibial nerve stimulation

External stimulation of the ankle with self-adhesive electrode pads. It is particularly effective for urge and faecal incontinence. Uses 10Hz

See Section 10.3. for more information about this programme.

12) TIBN 2 – Transcutaneous tibial nerve stimulation

Same as 11, but at 20 Hz.

For the programmes that cause muscle contractions, it is advisable not to 'overdo' it early on, as the resulting aches may not be felt until the next day. As long as you can feel the contraction, it is working. You can build up slowly over a number of days.

Most of the programmes cycle between **Work** and **Rest** to allow your muscles to recover in between contractions.

For best results in the programmes that cause muscle contractions try to contract the pelvic floor muscles along with the **Sure Pro**, and to sustain the contraction into the rest interval. If possible, link the contraction to your breathing to get into a gentle rhythm.

When you change the strength setting, this cycle stays on Work until you stop pressing the buttons Ch1▲ or Ch2▲ for more than 5 seconds.

Once the pelvic floor has been strengthened using the **Sure Pro**, continue to exercise the pelvic floor muscles regularly.

The uses mentioned above for each programme are indicative only, it is possible to adapt the use according to your needs.

Choosing the Right Strength

The goal of the programmes that cause muscle contractions is to produce powerful muscle contractions. The strength of the intensity should be increased to **about three times the level at which you can first feel the tingling**, or to as high as you can stand without causing pain. You will probably feel that electrical contraction is being more powerful than a voluntary contraction, because the current also stimulates your sensory nerves. The signals have a pain-relieving effect.

You may find the sensation uncomfortable to start with, so that you may not get up to therapeutic strength at the start of the treatment. The strength can be increased

during the course of the treatment, as you become accustomed to the sensation.

The powerful muscle contractions caused by electrical stimulation sometimes give rise to training aches, which usually disappear within a week.

Treatment Time and Treatment Interval

Current clinical evidence indicates that there should normally be no need to exceed the default treatment time settings in all but

the **URGE**, **URGE 2** and **PAIN** programmes. These may be used continuously if required.

Most clinical trials for stress incontinence (**STRE 1** and **STRE 2**) have used no more than one treatment per day. The evidence seems to show that there is no additional improvement to using stimulation more than 3 times a week. However, for home use, better long- term compliance is achieved by establishing a routine of using **Sure Pro** daily.

PROGRAMME SETTINGS

	Programme	Frequency (Hz)	Pulse Width (µs)	Ramp (sec)	Work (sec)	Rest (sec)	Default duration (min)
1	STRE 1	50	300	1	5	10	20
2	STRE 2	35	250	2	3	6	20
3	URGE	10	200	1	5	10	20
4	MIXED	10/50	200/300				30
5	URGE 2	10	200	Constant			Continuous
6	SENS	3/10/20/30/40	250/200				25
7	PFW A	See the table below					
8	PFW B	See the table below					
9	ENDUR	20	250	2	5	10	20
10	PAIN	4	200	Constant			20
11	TIBN 1	10	200	Constant			20
12	TIBN 2	20	200	Constant			20
13	CSTM 1	4-90	50-400	1-10	1-20	0-30	5-60
14	CSTM 2	4-90	50-400	1-10	1-20	0-30	5-60
15	CSTM 3	4-90	50-400	1-10	1-20	0-30	5-60

PFW	Frequency (Hz)	Pulse Width (μs)	Work (sec)	Rest (sec)	Default duration (min)	
A	20	250 μs fixed	4	6	2	
	20	250 μs stepped	4	6	5	
	3	250 μs fixed	4	4	5	
	10	500 μs fixed	4	4	15	
	20	250 μs stepped	4	6	10	
	35	250 μs stepped	4	8	5	
	45	250 μs stepped	4	8	5	
	10	500 μs fixed	4	4	5	
					Total	52 min
	B	20	250 μs fixed	6	8	2
20		250 μs stepped	6	8	5	
3		250 μs fixed	6	6	5	
10		500 μs fixed	6	6	15	
20		250 μs stepped	6	8	10	
35		250 μs stepped	6	12	5	
45		250 μs stepped	6	12	5	
10		500 μs fixed	6	6	5	
				Total	52 min	

In PFWA the Pulse Width increases from 175 μs to 250 μs in 4 secs - 25 μs per sec. This stepping up occurs during Output ON time.

In PFWB the Pulse Width increases from 170 μs to 250 μs in 8 secs - 10 μs per sec.

TRANSCUTANEOUS TIBIAL NERVE STIMULATION

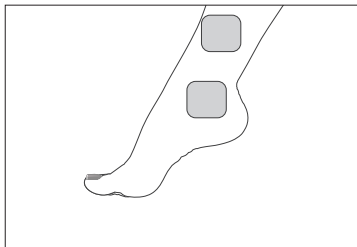
Transcutaneous stimulation of the posterior tibial nerve is effective for the treatment of urge and faecal incontinence.

The exact mechanism of action is unclear, but it is thought that tibial nerve contains mixed sensory motor nerve fibres that originate from the same spinal segments as the nerves to the bladder and pelvic floor.

Clinical trials have shown that TNS using a self-adhesive surface stimulation electrode without an implanted needle electrode (or internal electrode) can be effective.

(Ref: Joanne Booth et al. 2018. The effectiveness of transcutaneous tibial nerve stimulation (TTNS) for adults with overactive bladder syndrome: A systematic review. Neurology and Urodynamics. Volume37, Issue2. Feb 2018. 528-541)

Place the self-adhesive electrodes on the ankle skin with the negative (black lead wire connector) electrode behind the internal malleolus and the positive electrode (red lead wire connector) about 10 cm, above the negative electrode as shown below

Choose the programmes 11 or 12 (TIBN 1 or TIBN 2). Adjust intensity level until you see your toes moving, then reduce 1 or 2 mA. Use for 20 minutes once or twice daily.

The **Sure Pro** will send an electrical impulse to the nerve. This nerve impulse is then transmitted to the sacral plexus which regulates the control of bladder and pelvic floor muscles.

Warning: The **TIBN 1** and **TIBN 2** programmes are designed to be used only with self-adhesive electrode pads. Using these programmes with a vaginal probe could produce a too strong and unpleasant stimulation.

When you select the **TIBN 1** or **TIBN 2** programmes, the intensity freezes to 1, the warning triangle flashes and the buttons Ch1▲ and Ch2▲ are locked. Press ↵ to confirm the use of this programme or to change programmes.

MANUAL PROGRAMMES

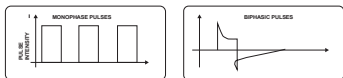
Warning: Consult your healthcare professional before using these programmes.

Correct settings depend on your muscle tone and exercise goals. Inappropriate settings could cause discomfort or muscle injury or aggravate Urge incontinence.

The programmes 13, 14 and 15 (respectively **CSTM 1**, **CSTM 2**, **CSTM 3**) are the manual programmes.

The effect of electrical stimulation on the body depends on a number of settings. In these programmes you can adjust the following:


Pulse waveform

This describes the time function of the excitation current which may be either monophasic or biphasic. With monophasic pulse trains, the current flows in one direction. With biphasic pulses, the excitation current alternates its direction.

The **Sure Pro** uses only biphasic pulse trains as they reduce the strain on the muscle, leading to less muscle fatigue as well as safer application, and reduce the risk of skin irritation under the electrode.

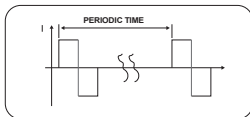Pulse frequency

Frequency indicates the number of individual pulses per second and is indicated in Hz (Hertz= pulses per second). It can be calculated by working out the inverse value of the periodic time.

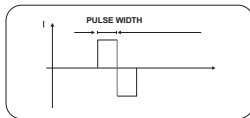
Different types of muscle fibres react preferentially to different frequencies: Slow-response fibres tend to react to lower pulse

frequencies up to 15Hz, while fast- response fibres only respond to frequencies over approx. 35Hz.

With pulses of 45 to 70Hz, there is permanent tension in the muscle (tetany) combined with premature muscle fatigue. Higher pulse frequencies can therefore be used for elasticity and maximum strength training.

For soothing and pain relief: A frequency of 90 Hz is good at blocking pain signals. A low frequency of 4 or 10Hz allows for the release of endorphins, the body's natural morphine-like substances.

Pulse Width


Pulse width is used to indicate the duration of an individual pulse in microseconds (μ s).

A larger muscle mass requires a greater pulse width. A higher Pulse Width is also more likely to activate pain nerves, so there is a fine balance between maximum muscle stimulation and tolerable sensation.

Muscle stimulation: 50–400 μ s.

Pain relief and soothing: 50 to 250 μ s.

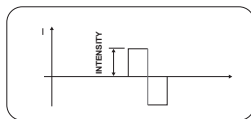Pulse Intensity

Setting the degree of intensity is dependent on the subjective feeling of each individual user and is determined by a number of parameters such as application site, skin circulation, skin thickness as well as quality of electrode contact. The actual setting should be effective but should never produce any unpleasant sensation such as pain at the site of application.

In pain relief and soothing programmes, a slight tingling sensation indicates sufficient stimulation energy. As you become accustomed to the stimulation, you may need to increase the intensity

In muscle stimulation programmes, the intensity needs to be as high as possible for maximum benefit – so set just below the pain threshold.

With prolonged application, you may need to increase intensity as nerves get used to the stimulation and become less sensitive (known as accommodation).

Ramp (rise)

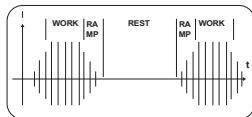
This is the time in seconds taken to move up and down between zero and the set stimulation strength. The **Sure Pro** can be adjusted from 1 to 10 seconds in steps of 0.5 sec.

Work

This is the time in seconds that muscle is stimulated (not including Ramp time). The **Sure Pro** offers a range of work periods from 1 to 20 sec.

Rest

This is the time in seconds at zero strength in between stimulation. The **Sure Pro** offers a range of rest periods from 0 to 30 sec. The EMS programmes use an Active Rest – low frequency pulses help to clear metabolites in between Work periods.

Note: If Rest is set to 0, Ramp is also zero and output is continuous.

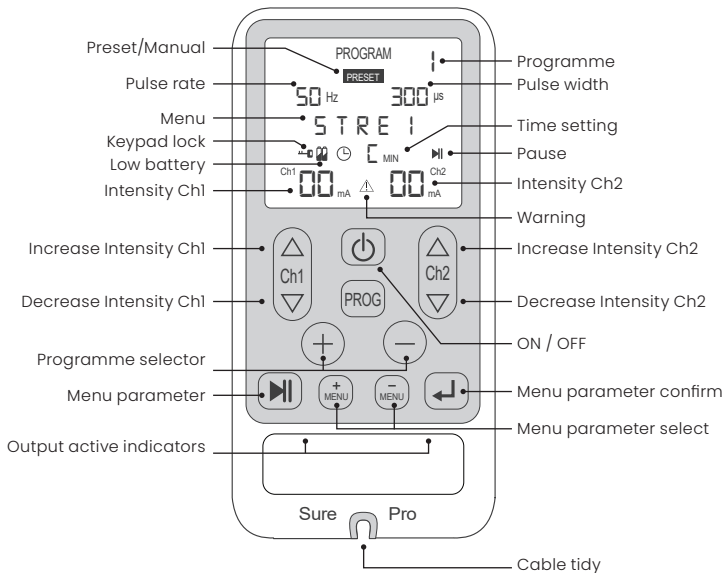
Warning: The warning triangle is displayed if the Rest period is less than:

$$\text{RestTime} = \text{Worktime} * (\text{WorkHz} - 16.66) / 16.66$$
seconds.

This is because muscle fibres can only activate a limited number of times a minute (about 1000) without becoming fatigued.

If the warning triangle is displayed at any other time, refer to troubleshooting section of this manual.

1.5. Controls and display

2. How to Use

2.1. Device setup

On/Off

1. To turn the unit on, press the ON/OFF button and hold for 3 to 5 seconds until the display shows.
2. To turn the unit off, press Ch1▼ or Ch2▼ button to unlock the keypad and press the ON/OFF button and hold for 3 to 5 seconds until the display stops.

At first use, the display shows that the unit is automatically set in programme STRE 1 at zero strength.

When switched on the unit will automatically start in the programme which was being used when it was last switched off.

The unit will turn off automatically:

- When the Timer reaches zero,
- If it is left at zero strength for more than 5 minutes.

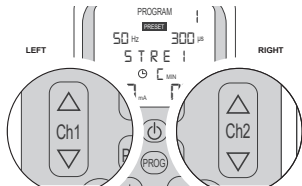
Note: Always check unit is OFF before applying or removing probe or pads.

The backlight will turn off 10 seconds after the last button press.

Strength controls

The buttons marked Ch1▲, Ch1▼, Ch2▲ and Ch2▼ are the strength controls.

The left-hand buttons change strength in the left-hand lead, Ch1, and the right-hand buttons change the right-hand lead, Ch2.

To increase strength in steps of 1 mA, press and release the top part of the two channel buttons, marked Ch1▲ and Ch2▲.

To decrease the strength, press and release the lower part of the buttons marked Ch1▲ and Ch2▲.

The unit will remain in the WORK part of the cycle for 5 seconds while intensity is being adjusted.

The strength levels are shown on the LCD.

The strength control buttons will not operate until the unit is properly connected to you (probe inserted correctly). **Sure Pro** detects a disconnection and automatically returns the strength to zero.

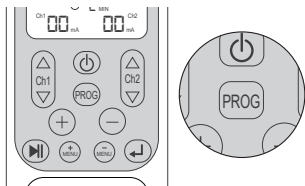
The unit has 99 levels of strength. If you hold down the button Ch1▲ or Ch2▲ for 3 to 5 seconds, the strength will start scrolling.

You may feel nothing over the first few presses. Continue pressing until the sensation is strong but comfortable. Further increases during use may be necessary if your body becomes used to the sensation.

The yellow LED on the output socket indicates that there is an active output. The display will remain on for 5 seconds after the plug is removed.

Programme select

The button marked PROG is the programme control. The **Sure Pro** has 12 preset programmes.

On first use the STRE 1 programme is automatically selected. On the next use it will start with the last programme used.

Each time you press and release the PROG button, the programme will change, and the screen will display the programme selected.

Each time you change the programme, the intensity will reset to zero. This safety feature prevents sudden unpleasant changes in stimulation, since each programme produces a different sensation.

The backlight turns off 5 seconds after the last button press.

Treatment timer

The default setting for preset programmes is shown on the screen.

To alter the Treatment Timer setting, press +Menu or -Menu. The display will flash. Use + and - buttons to adjust the treatment time. Then press ↵ to accept the change.

You can set session times of C (continuous) or 5 to 60 minutes in 5 minutes steps.

Programmes 4, 6, 7 & 8 are preset programmes with fixed treatment times.

Manual settings

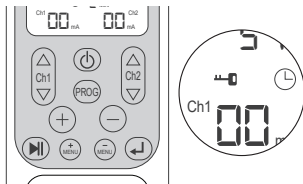
When a programme has manual settings available, MANUAL will be displayed on the screen.

Press +Menu or -Menu buttons to cycle through parameters. The selected parameter will flash. Press + or - to adjust the setting. Press ↵ to accept the change, and the flashing will stop.

Parameters cycle through Hz, μ s, WORK, REST, RAMP, TIMER, DATE, TIME.

Manual programme lock

When MANUAL is showing, you can protect the manual settings (or default settings) by pressing and holding ↵ button for 3 to 5 seconds. The lock symbol will appear on the screen and will flash if you try to adjust manual settings. To unlock simply press and hold button ↵ for 3 to 5 seconds again.

Usage memory

Date and time Setting:

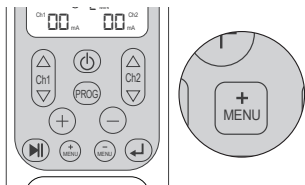
The date (Day of month) and time (Hour of day) can be set. This enables the memory to give an exact history of daily usage.

To set date and time, press +Menu and cycle through parameters to DATE.

Centre left shows Day of month and centre right shows Hour. Day is flashing. Adjust with + and - buttons and set with ↵ button.

To set Hour, press +Menu and cycle through parameters to TIME. Adjust with + and - buttons and set with ↵ button.

Memory Mode:

A: Day of month

B: Recording number for day

C: Programme used for this recording

D: Duration of recording

E: Average intensity for recording

If no programme has been used for more than 9 minutes, a NULL message is displayed, and the unit returns to waiting mode.

To view the memory, press +Menu or -Menu and cycle through parameters to the screen with MONTH showing in the middle and the warning triangle \triangle at bottom centre.

Use the + and - buttons to cycle through recordings.

To clear the memory, press and hold the +Menu and -Menu buttons together for 3 to 5 seconds while in memory mode.

Pause

Pressing the ►|| button while a programme is in use stops the stimulation and the timer. The symbol ►|| is displayed. Press again to resume the programme. Stimulation will re-start at 75% of previous intensity setting. If the device is left in PAUSE for more than 15 minutes, it automatically switches off.

Open circuit detector

Sure Pro controls the connections and the contact between the probe or electrodes and your body.

This is to avoid sudden changes in stimulation if a broken connection is re-established.

If the lead or probe does not have good contact or become detached and the intensity is set to greater than 10mA, the warning triangle symbol \triangle will flash, the zeroes in the display will flash, and the intensity will automatically reset to zero.

Check the lead and try using a water-based lubricating gel such as TensCare Go Gel (K-GO) with the probe.

To preserve battery life, the **Sure Pro** automatically switches off if left at zero intensity for more than 2 minutes.

Low battery

A low battery warning symbol will display when you need to recharge the battery.

Automatic keypad lock

If no button used for 10 seconds and intensity is set above zero the keypad will lock automatically. This prevents accidental changes to settings.

Press Ch1▼ or Ch2▼ button to unlock.

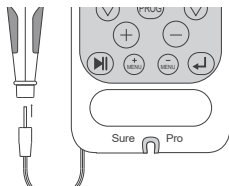
Connecting the probe

Decide whether to use one channel or two. Unless you are told otherwise by your medical advisor you will only want to use one. (Some treatment protocols use both vaginal and anal electrodes).

1. Insert the lead wire plug into the base of the unit.

2. Connect the lead from the base of the unit to the lead in the probe.

Push the pin ends firmly into the pigtail ends of the probe lead.

For **PAIN** and **TIBN**, use the self-adhesive electrode pads.

Note: The ends of the lead wire are coloured black or red. This coding is provided for some professional uses.

For most purposes, the orientation makes no difference, and you can ignore this colour coding.

Preparing for a session

1. Before using **Sure Pro** you will need to visit the toilet.
2. Lubricate the metal plates of the probe with a water-based lubricant (such as TensCare's Go Gel).
3. Choose a comfortable position, such as lying down on your bed on your side with your knees raised.

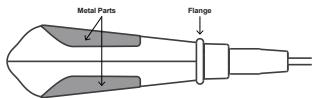
Inserting the probe

1. After the wire is securely connected, insert the probe into your vagina, in the same way as you would a tampon, until only the plastic rim at the base of the probe is visible.


Warning: Ensure the **Sure Pro** is switched off before inserting the probe.

2. Ensure the metal sides of the probe are facing toward your hips.

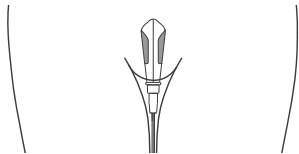3. Avoid inserting the probe too deeply; the flange at the end should remain outside the vagina for proper positioning.

Note: Wearing tight-fitting underwear or jeans can help keep the probe in place and maintain proper contact during the session.

2.2. Starting a session

Press and hold the ON/OFF button on the control unit for at least 3 seconds to switch the control unit on.

Programme selection

1. You can select from the pre-set programmes or a manual programme. Details in section 10 will help you identify the best programme to suit you.

2. To select a programme, press the PROG button.
3. With the required programme selected, you can adjust the intensity of the muscle stimulation with the ▼ and ▲ buttons until you reach a comfortable level. Once you have reached a comfortable level, 5 seconds after you stop pressing the button, the intermittent Work/Rest phase will start.

The machine will take itself to 0 mA for a Rest period and then take itself back up to the level of intensity you chose, to Work the muscle. This cycle will continue for the 20 minute programme.

Note: The strength required varies widely between users – some will use the **Sure Pro** at full power – 99 mA.

The **Sure Pro** strength will go up at 1 mA increments.

Initially the sensation through the probe may be limited but will improve during treatment. Take care not to use too much strength and thereby over stimulate the muscles until normal sensation is restored. The sensation may not be even as it may vary depending on the sensitivity of the nerves in the treatment area.

The display shows the strength of intensity used. The aim is to increase this over a number of days. But remember there is no hurry, so only increase the strength of the stimulation as and when you are comfortable and ready to progress.

i **Note:** If the sensation becomes uncomfortable, reduce the intensity with the Ch1▼ or Ch2▲ button.

i **Note:** If the Liberty probe does not make good contact with the walls of the vagina, you can try the Liberty Plus probe (X-VPM). This probe is 32mm diameter compared with the 28mm of the liberty probe.

The duration of most treatment sessions is automatically set to 20 minutes. The duration of muscle building sessions will depend greatly on your ability to contract and your fatigue resistance. Be careful not to overdrive the muscles, this could cause painful aches the next day.

i **Note:** If you have aching or cramping muscles, stop treatment until the symptoms go away, then resume at a lower intensity.

2.3. Completing a treatment

When the timer reaches zero, your session is complete and the unit turns off.

1. Check that the control unit is OFF. If it is not, hold down the ON/OFF button to switch off then remove the probe from your vagina by holding the positioning end rim and gently pulling outwards.
2. Wash and thoroughly dry the probe and return it to the storage pouch.
3. **Sure Pro** will not only improve your pelvic floor muscles but also help you to recognise the correct sensation you need to feel when doing your Kegel exercises.

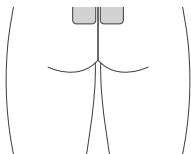
i **Note:** When removing the probe, DO NOT PULL ON THE LEAD WIRE.

2.4. Alternative electrode treatment

Optional electrode placement for URGE

An alternative method to a vaginal probe is to stimulate areas of the skin that are close to the nerves that go to the bladder and urethra. These come from the parts of the spinal cord segment called S2-S3. See electrode placement picture below.


The strength should be comfortable, but always remain noticeable. You may need to increase it over the course of the treatment.

Optional electrode placement for STRESS

The electrodes are placed on the skin between the anus and the genitals. See electrode placement picture below.

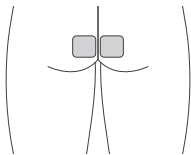The stimulation should be strong enough to make your anus contract slightly.

2.5. Alternative anal probe treatment

An anal probe, such as TesnCare X-PRI3 or X-AP can be purchased as an accessory. These probes can be used for urinary and faecal incontinence in both males and females.

These anal probes may be used in a similar way to the vaginal probe. Because the stimulation cannot be restricted to one muscle group, and the mucosal tissue has different electrical characteristics, anal stimulation is less comfortable than vaginal.

You should consult your healthcare professional before starting treatment.

Faecal Incontinence

Faecal incontinence can be the result of weakened or poorly functioning anal sphincter muscles or damage to the nerves controlling them. The purpose is to re-educate the anal sphincter and other muscles of the pelvic floor to contract. The treatments aim to progress towards graduated active exercises, in order to improve pelvic floor muscles' strength and endurance and to regain function.

You may benefit from the **Sure Pro** if you either have no active anal sphincter contraction, or a weak or poorly sustained contraction. Use the STRE 1 or STRE 2 programmes. Intensity should be as strong as possible without being painful. When possible, try to contract the muscles at the same time as the **Sure Pro**.

Post Prostatectomy Urinary Incontinence

Electrical stimulation has been found to help urinary incontinence in men after radical prostatectomy in some trials. Use the same programmes as for vaginal stimulation. Increase intensity to the highest tolerable.

How to insert the anal probe

1. Before using **Sure Pro** you will need to visit the toilet.
2. Lubricate the metal plates of the probe with a water-based lubricant (such as TensCare's Go Gel).

Caution: Do not use a silicone based lubricant on the stimulation contacts as it may decrease the effectiveness of the **Sure Pro**'s muscle stimulation.

3. Choose a comfortable position, such as lying down on your bed on your side with your knees raised.

Warning: Ensure the **Sure Pro** is switched OFF before inserting the probe.

4. After wires are securely connected, insert the probe into the anus whilst 'bearing down' (as in the action of passing stool) to a comfortable limit until the base of the flange on the probe touches the anus. The metal parts conduct the electrical pulse and should be in contact with the main part of the muscle at all times. The tissues close to the entrance are more sensitive, so you should avoid stimulating them. It is recommended that the probe is inserted past the sphincter muscles of

the anus, unless directed otherwise by a healthcare professional.

5. Anal probes with long electrodes (the metal part) that run up and down the length of the attachment should always be inserted with the metal parts facing hip-to-hip. Anal probes with circular electrodes (the metal part) should be inserted simply to the desired depth.
 - For Faecal incontinence, the aim is to stimulate the external sphincter and/or pubo-rectal muscle, so circular electrodes should be placed so that the external ring is just inside the sphincter.
 - For Urinary Stress incontinence the aim is to stimulate the levator muscles and the probe should be inserted deeper.

2.6. Kegel exercises

In addition to electrode therapy, performing Kegel exercises can significantly improve bladder control. Kegel exercises involve repeatedly contracting and relaxing the pelvic floor muscles, which can strengthen these muscles and help manage both stress and urge incontinence. Regular practice of Kegel exercises is recommended for optimal results.

1. **Identify the right muscles:** To locate your pelvic floor muscles, try stopping urination midstream. The muscles you use for this are your pelvic floor muscles.
2. **Get comfortable:** You can do Kegel exercises in various positions—lying down, sitting, or standing. Choose a position that feels comfortable for you.

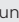
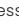
3. **Contract the muscles:** Tighten your pelvic floor muscles and hold the contraction for about 5 seconds.
4. **Relax:** Release the contraction and relax the muscles for 5 seconds. Make sure to breathe normally throughout the exercise.
5. **Repeat:** Aim for 10-15 repetitions per session. You can gradually increase the hold time and the number of repetitions as you get stronger.

Tips

- Avoid using your abdominal, leg, or buttock muscles while doing Kegels.
- Do not perform Kegels while urinating, as this can lead to incomplete emptying of the bladder over time.
- Be patient; it may take a few weeks to notice improvements.
- Perform Kegel exercises three times a day for the best results.

3. Troubleshooting

Problem	Possible causes	Solution
No display	Flat battery	Charge battery. See section 13.1.
	Damaged battery	Contact supplier.
Low battery display	Low battery	Charge battery.
Controls don't work	Key pad is locked	If  is shown on display, press Ch1▼ or Ch2▼ button to unlock the keypad. If  is not showing, check that battery is charged.
Warning triangle flashing	Awaiting confirmation or wrong programme	If you are in programme TIBN 1 or TIBN 2 and you have attached the self-adhesive electrode pads to your ankle, press  button to activate the programme. Otherwise, select a different programme.
Intensity won't go above 10mA and zeroes in display and warning triangle flashing	The Sure Pro has a safety feature which will not allow the intensity to pass 10mA if the machine detects a connection error. If a connection error is detected the intensity will return to 0 mA and the warning triangle will flash. This safety feature will prevent the machine from giving any uncomfortable stimulation should the contact break between the machine and your skin. This will also prevent anyone from increasing the intensity to a high level without firm contact between the machine and the skin. A connection error can occur if:	

1. A break has developed within one of the two lead wires.

i) If this happens, check that the lead wire is correctly connected. (If you are using the electrode pads, ensure that they are correctly attached to the skin).

ii) If you are using the probe, you can try to test the unit by holding the probe in your hand: Dampen your hand with water and a little table salt. Squeeze the probe firmly and make sure your skin is covering the metal parts of the probe and carefully increase strength until you can feel something. Most people will start to feel the stimulation in their hand at around 25 mA.

iii) If the warning triangle flashes and the unit will not allow you to pass 10 mA, the lead wire may need to be replaced. Try again using the other lead.

If you have tried the test above and DO have sensation when the probe is in your hand but not when inserted vaginally, then it may be that:

2. The skin is dry, meaning poor conductivity between the metal plates on the probe and your skin.

If this happens, you can try these solutions:

i) Using a water-based lubricant such as TensCare Go Gel (see K-GO), which will improve conduction.

ii) Crossing your legs and squeezing to increase pressure on the probe, which should improve the connection. If this enables you to use the unit, you should find that in a few weeks of stimulation the contact improves. If it does not, this unit may not be suitable for you. You may need to contact your healthcare professional to discuss other suitable options.

iii) The probe supplied with the unit has a 28 mm diameter. An optional 32 mm probe, part no. XVPM, is available.

No sensation and no warning triangle showing	Intensity level is not high enough and/or reduced sensitivity in the area being treated.	<p>i) Please make sure you are increasing the intensity high enough. Most people will start to feel the stimulation in their hand at around 25 mA and with the probe inserted you will need to increase the intensity higher to around 40 mA to 60 mA. Max power is 99 mA. Everyone is different so just keep increasing the intensity until you can feel it. The intensity increases in very small steps of 1 mA.</p> <p>ii) You may have reduced sensitivity due to previously damaged or desensitised pudendal nerves (this can happen in childbirth or some surgical procedures). Please consult your healthcare professional.</p>
--	--	--

No sensation on one side of the probe (or electrode)	Position is not optimal – needs adjusting.	The current flows from one side of the probe to the other, so it is not possible to have one side “not working”. However, the strength of the sensation depends on how close to the nerve the current flows, and also in which direction it flows relative to the nerve. You can try slightly adjusting the position on the probe or exchanging the connection of the wires in the probe.
--	--	---

Sudden change in sensation	If you disconnect and re-connect a few minutes later, the signal will feel quite a lot stronger.	Always return strength to zero after disconnecting the lead or the probe.
----------------------------	--	---

If you are unable to resolve the issue, please contact our customer service team:

Email: customer.service@tenscare.co.uk

Phone: +44 1372 723434

Note: Unicare has no essential performance characteristics. Device failure will not cause unacceptable clinical risk for the user.

4. Device Care

4.1. Cleaning

It is important that the probe is cleaned before using for the first time and after each use.

1. Clean with either an alcohol-free antibacterial wipe such as TensCare wipes or by washing with warm soapy water.

Warning: Do not immerse the probe in a liquid.

2. Rinse in clean water and dry thoroughly.
3. Return the unit to the storage pouch.

Clean the device and lead wires at least once a week by wiping with a damp cloth and solution of mild soap and water. Wipe dry.

- Do not immerse the device in water.
- Do not use any other cleaning solution than soap and water.

4.2. Storage

Between uses it is important to store the electrodes in their resealable plastic bag with the bag sealed. This will help extend the life of the electrodes.

Keep the device and all accessories in the storage pouch between uses.

Storage life

- The device has no fixed shelf life.

- Unused batteries have a nominal shelf life of 3 years but will usually last longer than this.

4.3. Charging Sure Pro

The **Sure Pro** is powered by a built-in rechargeable Li-ion battery. A power adaptor with interchangeable plug is included in the kit. The battery should need charging about once a month. The battery should last at least 15 hours at 50mA 300µs 50Hz. To fully charge the battery it will take about 2 hours.

When the battery is running low, the symbol will show on the screen. Although the display fades as the batteries run down, the strength of the output does not change until the warning is shown.

Charging the battery

1. Connect the charger to the unit and plug it onto the mains socket.
2. The word BATT and three flashing lines will show on the screen display – that means the battery is being charged.
3. When the battery is charged, the word FULL will display

Note: The device should be charged at least once every 3 months to ensure the battery life is maintained.

Note: The device cannot be used while being charged.

Note: Charging voltage: DC 5V, charging current: 90mA.

Battery life and maintenance

The battery should last at least 15 hours at 50mA, 300µs, 50Hz.

To fully charge the battery it will take about 2 hours.

Note: The device should be charged at least once every 3 months to ensure the battery life is maintained.

Note: Charging voltage: DC 5V, charging current: 90mA.

Battery Warnings

- Remove batteries from the device if it is unlikely to be used for a long period. Some types of batteries may leak corrosive fluid.
- Do not pierce, open, disassemble, or use in a humid and/or corrosive environment.
- Do not expose to temperatures over 60°C(140F).
- Do not put, store or leave near sources of heat, in direct strong sunlight, in a high temperature location, in a pressurized container or in a microwave oven.
- Do not immerse in water or get wet.
- Do not short-circuit.
- Do not connect the device unless the battery cover is in place.
- If battery leakage occurs and comes in contact with the skin or eyes, wash thoroughly with lots of water and immediately seek medical attention.

- Keep batteries out of the reach of children to prevent them from swallowing them by mistake. If swallowed by child, contact doctor immediately.
- Never attempt to recharge an alkaline battery. Risk of explosion.
- Do not mix old, new or different types of batteries as this may lead to battery leakage or low battery indication.
- Disposal: Always dispose of batteries and device responsibly according to local government guidelines. Do not throw batteries onto a fire. Risk of explosion.

4.4. Probe and lead wire care

Caution: The stainless steel in the probe's metal plates can contain some Nickel. This could cause a reaction if you have a Nickel allergy.

- Replace the probe every 6 months to maintain proper hygiene.
- Do not pull on the lead wires of the probe. This may cause damage.
- The probe is intended for single patient use only. Do not share the probe with anyone else. Improper treatment or cross-infection may occur.
- It is important the probe is cleaned after each use. Ineffective cleaning may lead to irritation or infection.
- To ensure safety and effectiveness we recommend using TensCare probes and electrodes only.

- The life of the lead wires depends greatly on use. Always handle the leads with care. We recommend replacing the lead wires regularly (about every 6 months).

4.5. Electrode care

The electrodes are reusable but for single patient use. The electrodes should provide up to 20 uses. This may vary depending on skin condition, humidity and storage between uses.

To maximise the life of the electrodes, always ensure the electrodes are replaced on to their plastic liner and stored and sealed in the plastic bag. When the electrodes dry, this may negatively impact the performance of the treatment.

The adhesive is the electrode is a peelable hydrogel (water based). To restore the tackiness of the electrode, add a tiny drop of water and spread it around. Avoid adding too much, as it will make the electrode too soft. If this occurs, place the electrodes sticky side up in a refrigerator for a few hours to regain adhesiveness.

Replace the electrodes when they lose their stickiness. Poor connection may cause discomfort and skin irritation.

To ensure safety and effectiveness we recommend using TensCare electrodes only.

4.6. Replacement parts

The following replacement parts are compatible with the Unicare and may be ordered from TensCare at www.tenscare.co.uk or +44(0) 1372 723434.

X-VP	Liberty vaginal probe (28mm diameter)
X-VPL	Liberty loop vaginal probe (32mm diameter)
X-VPM	Liberty plus vaginal probe (32mm diameter)
X-AP	Liberty mini vaginal probe (24mm width)
X-VPLG	Gold liberty loop vaginal probe (32mm diameter)
X-PR13	Liberty fit anal probe
E-CM5050	50x50mm electrodes (x4)
L-CPT	Lead wire
X-BX-UR	Belt clip
X-UNICH	Charging adapter
X-MULTI-UK	UK socket
X-MULTI-EU	EU socket
X-MULTI-USA	US socket


 **Warning:** Do not use this device with any probe other than those recommended above. Probes with smaller surface area may cause tissue irritation.

5. Technical Specification

Waveform	Symmetrical rectangular
Max Intensity	99mA zero to peak. Setting 0-99 in steps of 1 OC cut-out below 160 Ohm Constant voltage 470 to 2000 Ohm Constant current 160 to 470 Ohm
Channels	Dual channel
Max Pulse energy	Total output limited to 25 μ C per pulse
Output plug	Fully shielded: touch proof
Power	Built-in Li-ion polymer battery Ratings: 900 mAh, 3.33Wh, 3.7V
Battery life	At least 10 hours at 50 mA 300 μ s 50 Hz
Weight	90 g without batteries
Dimensions	115 x 56 x 23mm
Safety classification	Internal power source
Environmental Specifications	
Operating conditions	Temperature range: 5 to 40°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 70 kPa to 106 kPa
Storage conditions	Temperature range: -10 to +60°C Humidity: 15 to 75% RH non-condensing Atmospheric pressure: 70 kPa to 106 kPa

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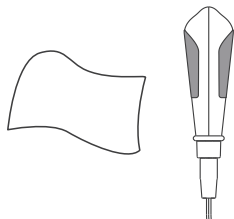2.6. Kegelübungen

In addition to electrode therapy, performing Kegel exercises can significantly improve bladder control. Kegel exercises involve repeatedly contracting and relaxing the pelvic floor muscles, which can strengthen these muscles and help manage both stress and urge incontinence. Regular practice of Kegel-Übungen werden für optimale Ergebnisse empfohlen.

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Thank you for using Sure Pro...

At TensCare, we strive to provide the best possible products and services. We value your feedback and continuously work to enhance our offerings. Understanding how our products benefit you is important to us. If you have any insights or suggestions, please get in touch.

Contact us

-  TensCare Ltd.
9 Blenheim Road,
Epsom, Surrey,
KT19 9BE, U.K.
-  +44 (0) 1372 723434
-  www.tenscare.co.uk
-  customer.service@tenscare.co.uk

Follow us

-  TensCare Ltd
-  @tenscareltd
-  @TensCareLtd
-  TensCare Limited

EU Declaration of Conformity

Tenscare Ltd herewith declare that the product meets the provisions of REGULATION (EU) 2017/745 which apply to it. The medical device has been assigned to class IIa according to Annex IV of the REGULATION (EU) 2017/745. The product concerned has been designed and manufactured under a quality management system according to Annex IX, of REGULATION (EU) 2017/745. This EU declaration of conformity is issued under the sole responsibility of the manufacturer. No "Common Specification" is applicable.

www.tenscare.co.uk

TensCare Ltd.

9 Blenheim Road, Epsom,
Surrey, KT19 9BE, U.K.

EU DoC

Complies with Medical Device
Regulation (EU) 2017/745.

Advena Ltd.

Tower Business Centre,
2nd Flr, Tower Street, Swatar,
BKR 4013, Malta

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