 unicare

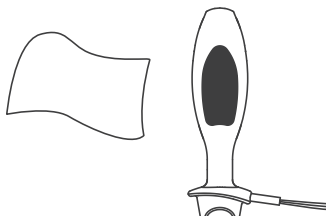
Instructions for Use  
Please read carefully

*TensCare™*

# Quickstart Guide


1

Clean the probe with a wipe or with warm soapy water.


2

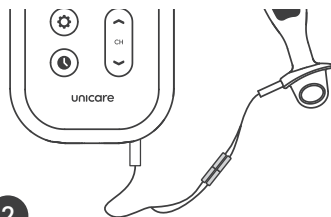Connect the probe to the device.


3

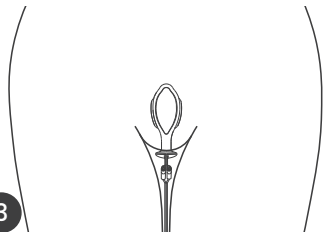Insert the probe into the vagina with the metal plates facing outwards.


4

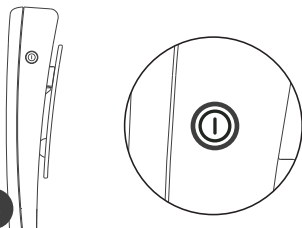Press and hold the power button to turn the device on.


5

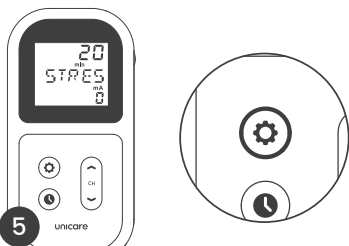Select a programme by pressing the programme selection button.

6

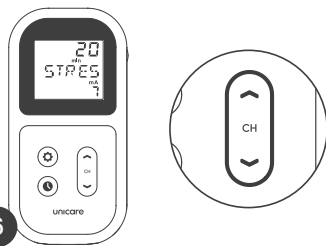Start the session by increasing the intensity.

# Contents

<b>1.</b>	<b>Device Description</b>	<b>3</b>
1.1.	Introduction	3
1.2.	Intended purpose	3
1.3.	Kit contents	3
1.4.	Treatment programmes	3
1.5.	Controls and display	5
<b>2.</b>	<b>How to Use</b>	<b>6</b>
2.1.	Device setup - Incontinence treatment	6
2.2.	Device setup - Period pain treatment	7
2.3.	Completing a treatment	7
2.4.	After a treatment	10
2.5.	Treatment guidance summary	12
2.6.	Alternative electrode treatment	12
2.7.	Alternative anal probe treatment	13
2.8.	Kegel exercises	13
<b>3.</b>	<b>Troubleshooting</b>	<b>14</b>
<b>4.</b>	<b>Device Care</b>	<b>15</b>
4.1.	Cleaning	15
4.2.	Storage	16
4.3.	Charging Unicare	16
4.4.	Probe and lead wire care	17
4.5.	Electrode care	17
4.6.	Replacement parts	17
<b>5.</b>	<b>Technical Specification</b>	<b>18</b>

# 1. Device Description

## 1.1. Introduction

Unicare is a clinically proven electrical muscle stimulator designed to treat urinary incontinence and improve your pelvic floor health. It uses gentle electrical signals delivered through a small vaginal probe to stimulate and contract the pelvic floor muscles. This promotes muscle strength and pelvic floor muscle and bladder control, helping to reduce or eliminate urine leakage. Pelvic floor stimulation may also help to enhance your sexual health.

Unicare can also be used to provide relief from period pain.

When used as intended, the Unicare can provide the following clinical benefits:

- Reduced frequency of stress incontinence episodes
- Reduced frequency of urge incontinence episodes
- Improved pelvic floor muscle strength and bladder control
- Relief from pain associated with primary dysmenorrhoea

## 1.2. Intended purpose

Unicare is a medical device designed to be used in the home healthcare environment to treat symptoms of stress, urge and mixed urinary incontinence as well as symptoms of primary menstrual pain (dysmenorrhoea).

Unicare may also help to relieve the pain of

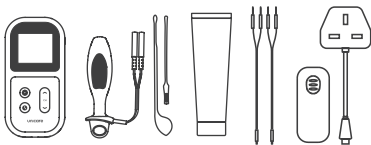
secondary menstrual pain due to conditions such as endometriosis, fibroids or pelvic inflammatory disease.

Unicare is suitable for use by adults (18 years+) who can control the device and understand the instructions.

Do not use the device for any other purpose.

## 1.3. Kit contents

- 1 x Unicare device
- 1 x Liberty Loop vaginal probe (X-VPL)
- 2 x 50x90mm electrodes (E-CM5090)
- 1 x Go Gel
- 1 x Lead wire (L-CPT)
- 1 x Detachable belt clip
- 1 x Charging adapter
- 1 x Storage pouch
- 1 x Instructions for Use

## 1.4. Treatment programmes

Unicare can be used to treat incontinence and alleviate period pain.

### Incontinence

Unicare has different Electrical Muscle Stimulation (EMS) treatment programmes depending on the type of incontinence you have.

## Stress Incontinence

Stress incontinence is when urine leaks during activities like coughing, sneezing, laughing, or exercising. This happens because the muscles supporting the bladder and urethra weaken, often due to childbirth, aging, or surgery. As a result, the muscles can't control urine release, leading to accidental leakage.

## Urge Incontinence

Urge incontinence is when there is a sudden, intense urge to urinate, followed by involuntary urine leakage. It occurs when the bladder muscles contract too often or uncontrollably, even when the bladder isn't full. This can be caused by infections, neurological conditions, or other underlying health issues.

## Mixed Incontinence

Mixed incontinence is when a person has both stress incontinence and urge incontinence, meaning they leak urine during activities like coughing or exercising and also have sudden, strong urges to urinate that they can't control.

## Period pain

Unicare's Pain programme uses Transcutaneous Electrical Nerve Stimulation (TENS) to alleviate symptoms of primary and secondary menstrual pain or period pain. Unicare's pain programmes stimulates sensory nerves to block pain signals from being received by the brain, providing instant and effective relief whilst being used. TENS is a clinically proven and safe treatment option for providing pain relief.

Select from the following treatment programmes:

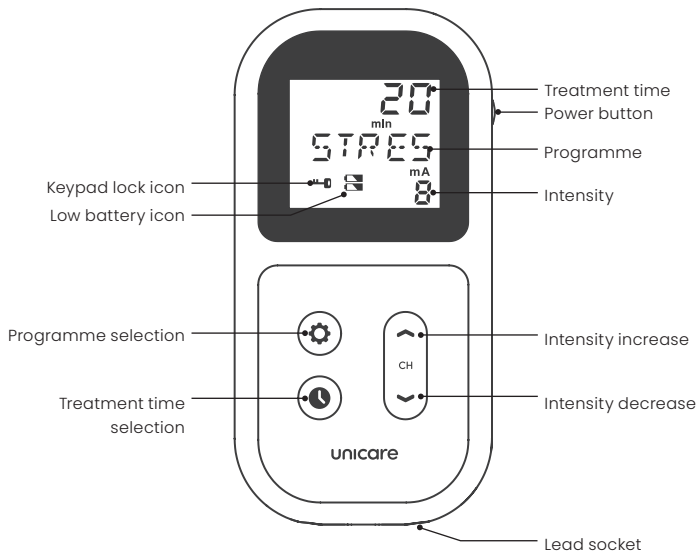
Prog.	Description	Default treatment time	Usage guidance
Stress	The stimulation triggers muscle contractions, which helps to strengthen them.	20 minutes	Use the device once daily, with noticeable improvements typically occurring within four weeks.
Urge	The gentle stimulation soothes the bladder muscle, reducing involuntary contractions and preventing unexpected bladder emptying.	Continuous	Use the device once daily, with noticeable improvements occurring in as little as two weeks.
Mixed	Combination of stress and urge programmes to treat mixed incontinence.	20 minutes	Use the device once daily, with noticeable improvements typically occurring within four weeks.

Tone	After-care programme for maintaining pelvic floor health once improvements have been made. This programme is also recommended for sexual health.	20 minutes	Use twice per week.
Pain	TENS stimulation provides relief from menstrual/period pain	Continuous	Use when experiencing pain.

**i** **Note:** Unicare incontinence treatment is most effective when complemented by Kegel Exercises. These involve voluntary contractions and relaxing of the pelvic floor muscles. See section 2.8 for guidance on performing Kegel Exercises.

## 1.5. Controls and display

## 2. How to Use

### 2.1. Device setup - Incontinence treatment

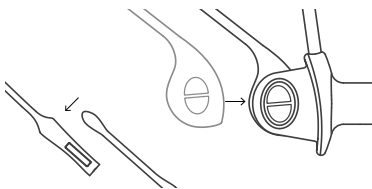
#### Connect the leads

1. Insert the plastic end of the lead wire into the socket on the rear of the device.
2. Push the metal pins of the lead wires into the sockets on the probe lead.

#### Connecting the pointer

The Liberty Loop probe features a detachable pointer, which serves as a visual indicator for monitoring the effectiveness of your pelvic floor muscle contractions.

1. Assemble the two halves of the pointer.
2. Insert the base of the pointer into the circular hole at the end of the probe, ensuring the pointer is positioned at the following angle:

#### Preparing for a session

1. Before inserting the probe, it is advised to empty your bladder.
2. Clean the probe before using for the

first time and after each use. Clean with either an alcohol-free antibacterial wipe such as TensCare Wipes or by washing with warm soapy water.

3. Lubricate the metal plates of the probe with the Go Gel.
4. Get into a comfortable position, such as sitting down or lying down with your knees bent and your feet flat.

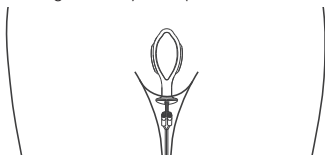
#### Inserting the probe

1. With the probe connected to the device and adequately lubricated, gently insert it into the vagina.


**Warning:** Ensure the Unicare is switched off before inserting the probe.

2. Ensure the metal sides of the probe are facing toward your hips.

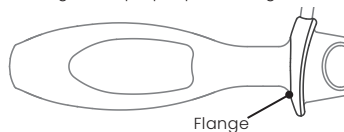3. Avoid inserting the probe too deeply; the flange at the end should remain outside the vagina for proper positioning.

**i** **Note:** Wearing tight-fitting underwear or jeans can help keep the probe in place and maintain proper contact during the session.

## 2.2. Device setup – Period pain treatment

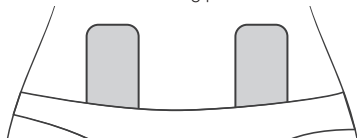
### Connect the leads

Insert the white, plastic end of the lead wire into the socket on the rear of the device.


Push the metal pins of the lead wires into the sockets on the electrodes.

### Position the electrodes

1. Before applying the electrodes, make sure your skin is clean and dry
2. Remove the electrodes from the plastic liner.
3. Apply the electrodes to the body, in one of the three following positions.

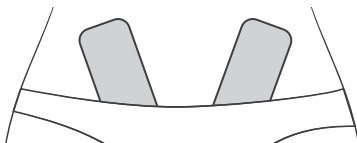
On the front in a vertical position about 10cms apart.

On the front in a diagonal position about

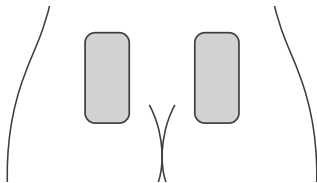10cms apart at the lower end of the electrodes.

On the back in a vertical position, about 10cms apart, with the bottom of the electrodes no higher than the top of your buttocks.

**i** **Note:** Always ensure the device is turned off or the intensity is zero before applying the electrodes.

**i** **Note:** TENS stimulation will work differently depending on the location of your nerves. Therefore, it is important to try all these positions to find what provides you with the most pain relief.

## 2.3. Completing a treatment

### Turning on / off

Turn the device on and off by pressing the power button for 3 to 5 seconds.

**i** **Note:** The device will automatically turn off if the intensity setting is left at zero for more than 5 minutes.

## Programme selection

1. Select the desired programme by pressing the programme selection button. (See section 1.4 Treatment programmes).

Each time you change the programme the intensity level will revert to zero.

When turning on, the device will remember the last programme used.

## Treatment time selection

1. To select the treatment time, press the treatment time selection button. The min icon will flash on the display.

2. Press ▲▼ to change the treatment time from either 10, 20, 30, 45, 60, 90 minutes or continuous.
3. Press treatment time selection button again to confirm the treatment time.

It is recommended to use the default treatment times for each programme.

**Note:** As with regular muscle exercise, the longer the training session the greater the muscle fatigue and time for recovery.

**Note:** Treatment time cannot be adjusted in the Mixed programme. This is set at 20 minutes.

**Note:** For Urge programme, the default treatment time is set as continuous. Start with a daily treatment of 20 minutes and increase the duration if you do not notice improvements after 2 weeks.

**Note:** The pain programme can be used for as long as necessary. While you can use it for extended periods, maximum relief is typically achieved after about 40 minutes. Longer use may cause muscle ache.

## Starting a session

With the probe inserted or electrodes connected, start the treatment session by increasing the intensity with the ▲ button.

The treatment countdown timer will begin.

## Intensity selection

Select the stimulation intensity by using the ▲▼ buttons.

Unicare has 99 intensity levels. Each press of the ▲ and ▼ buttons will adjust the intensity by 1mA. The intensity can be increased or decreased quickly by pressing and holding the ▲▼ buttons.

Prog.	Recommended intensity setting
Stress	Increase the intensity until the muscles around the vagina contract.

<b>Urge</b>	Increase the intensity such that it is strong but comfortable. No muscle contractions are needed.
<b>Mixed</b>	First 10 minutes: See Urge programme guidance. Second 10 minutes: See Stress programme guidance.
<b>Tone</b>	Increase the intensity until the muscles around the vagina contract.
<b>Pain</b>	Increase the intensity such that it is strong but comfortable. No muscle contractions are needed.

Intensity may need to be increased mid-way through treatment as your body gets used to the sensation.

If the sensation becomes uncomfortable, reduce the intensity.

**Note:** At intensity levels about 20, if the device detects a disconnection it will automatically return the intensity to zero, the display will flash "LEADS" and the device will beep three times.

To increase intensity, ensure the probe is properly inserted or the electrodes are properly connecte and the lead wires are securely connected to both the probe and the device.

**Warning:** When the yellow LEDs in the channel output sockets light, this indicates the stimulation intensity is

high enough to be harmful if not used correctly. Please use with caution.

**Note:** If you experience muscle cramping, pause the treatment until the symptoms subside. Then, resume the session at a lower intensity.

### Stimulation output

For the Stress and Tone programmes, the stimulation is intermittent.

The programmes have 'work' periods in which the stimulation is active and 'rest' periods with no stimulation.

This pattern ensures effective muscle engagement and adequate recovery time between contractions.

During the 'rest' periods, the intensity setting on the display will flash 00.0. The rest period will last between 5-10 seconds depending on the selected programme, before returning to the 'work period'.

During the 'work' period, the display will show the selected intensity setting, and you should feel the stimulation.

For the best results, try to voluntarily contract

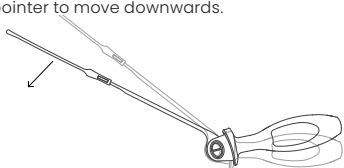
the pelvic floor muscles during the work periods and sustain the contractions into the rest periods.

The Urge programme and second half of the mixed programme have a constant output and do not require you to voluntarily try and contract your muscles.


### Using the pointer

The probe pointer will give an indication of the strength and duration of each contraction, and whether or not the contraction is being performed correctly.

Correct muscle contractions will cause the pointer to move downwards.

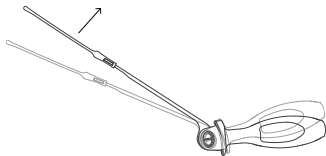Incorrect muscle contractions will cause the pointer to move upwards.

Little or no movements indicates weak muscle contractions.

### Keypad lock

If you do not press any keys for 30 seconds, the keypad will lock. This is to avoid

accidental changes in setting. To unlock, press the ▼ button.

### Low battery

When the battery is low, the battery icons will display on the screen.

The device will automatically turn off after 2 minutes.

Please charge your device.

## 2.4. After a treatment

### Disconnecting the device

Once the treatment timer reaches zero, the treatment session will end, and the device turns off automatically.

1. Remove the probe by holding the flange and gently pulling outwards or remove the electrodes from the skin by holding the pad and gently pulling.

**Warning:** Make sure the device is off before removing the probe/electrodes.

**Note:** Do not pull on the lead wire to remove the probe, as this can cause damage.

2. Replace the electrodes on the plastic liner.

3. Disconnect the leads from the probe/ electrodes and the device.
4. Place the electrodes in the resealable plastic bag.
5. Place the leads and the device in the storage pouch.

### Cleaning the probe

It is important the probe is cleaned after every use.

1. Clean the probe with either an alcohol-free antibacterial wipe or by washing with warm soapy water.
2. Rinse in clean water and dry thoroughly.
3. Place the probe in the storage pouch between uses.

### Frequency of use

Prog.	Frequency of use	Time to improvement
<b>Stress</b>	Once per day	Improvements may be noticeable after 4 weeks
<b>Urge</b>	Once per day	Some improvements may be noticeable after 2 weeks
<b>Mixed</b>	Once per day	Improvements may be noticeable after 4 weeks
<b>Tone</b>	Twice per week	N/A – programme should be used to maintain pelvic health
<b>Pain</b>	Whilst experiencing pain	Programme should provide immediate relief during use.

**Note:** Time to improvements will vary from person to person.

### Usage memory

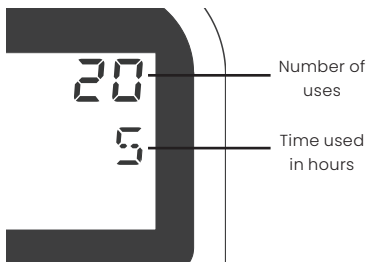
The Unicare has a memory function.

To view the usage memory, press T and ▼ buttons for 3-5 seconds.

The display will show the number of times used and duration in hours used.

Press the same buttons again to return to normal controls.

To reset memory to zero, hold down the T and off buttons together for 3 to 5 seconds.


## 2.5. Treatment guidance summary

	Aim	Treatment time	Intensity	Frequency of use
<b>Stress</b>	Treat stress incontinence	20 minutes	Increase until muscles contract	Once per day
<b>Urge</b>	Treat urge incontinence	Default continuous. Start with 20 minutes and increase the duration if you do not notice improvements.	Increase until strong but comfortable. No muscle contractions are needed.	Once per day
<b>Mixed</b>	Treat mixed incontinence	20 minutes	First 10 minutes: See Urge guidance Second 10 minutes: See Stress guidance	Once per day
<b>Tone</b>	After-care for maintaining pelvic floor health	20 minutes	Increase until muscles contract.	Twice per week
<b>Pain</b>	Relief from menstrual/period pain	As long as necessary to provide pain relief	Increase until strong but comfortable. No muscle contractions are needed.	Whilst experiencing pain

## 2.6. Alternative electrode treatment


Alternatively, stress and urge incontinence can be treated using skin electrodes, though this method may be a less effective treatment than using a vaginal probe.

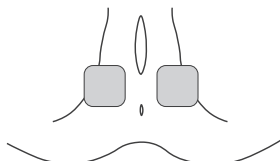For stress incontinence, place the electrode on the skin between the anus and genitals.

Ensure the stimulation is strong enough to cause a slight contraction of the anus.

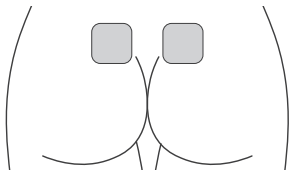For urge incontinence, place the electrode either side of the spine, close to the buttocks.

The stimulation should be strong yet comfortable.

Follow the treatment time guidance provided for vaginal probe treatment.

## 2.7. Alternative anal probe treatment

The TensCare X-PR13 anal probe is available for purchase and can be used to treat faecal incontinence in both men and women.

Similar to how a vaginal probe is used, this device helps manage faecal incontinence by stimulating the external anal sphincter and/or the puborectalis muscle.

To achieve optimal results, ensure that the circular electrodes are positioned just inside the sphincter.

For treatment, use the STRES or TONE programs, and adjust the intensity to the highest level that remains comfortable and pain-free. Whenever possible, try to contract the pelvic floor muscles simultaneously to enhance the therapy's effectiveness.

## 2.8. Kegel exercises

In addition to electrode therapy, performing Kegel exercises can significantly improve bladder control. Kegel exercises involve

repeatedly contracting and relaxing the pelvic floor muscles, which can strengthen these muscles and help manage both stress and urge incontinence. Regular practice of Kegel exercises is recommended for optimal results.

1. **Identify the right muscles:** To locate your pelvic floor muscles, try stopping urination midstream. The muscles you use for this are your pelvic floor muscles.
2. **Get comfortable:** You can do Kegel exercises in various positions—lying down, sitting, or standing. Choose a position that feels comfortable for you.
3. **Contract the muscles:** Tighten your pelvic floor muscles and hold the contraction for about 5 seconds.
4. **Relax:** Release the contraction and relax the muscles for 5 seconds. Make sure to breathe normally throughout the exercise.
5. **Repeat:** Aim for 10-15 repetitions per session. You can gradually increase the hold time and the number of repetitions as you get stronger.

### Tips

- Avoid using your abdominal, leg, or buttock muscles while doing Kegels.
- Do not perform Kegels while urinating, as this can lead to incomplete emptying of the bladder over time.
- Be patient; it may take a few weeks to notice improvements.
- Perform Kegel exercises three times a day for the best results.

### 3. Troubleshooting

If you are experiencing issues with the device, refer to the troubleshooting guide below.

Problem	Possible causes	Solution
Display will not turn on	Flat batteries.	Recharge the device (see section 4.3).
	Damaged battery	Contact supplier. There is no maintenance or calibration other than replacement of gel pads and battery charging. Do not modify the device as this may reduce safety or effectiveness.
Low battery display	Low batteries.	Recharge the device.
Controls will not work	The keypad is locked.	If the key icon is shown, press the ▼ button to unlock the device.
No stimulation sensation / LEADS showing on the display	There is a connection error. This should be noticeable by LEADS displaying on the screen. This may be due to a number of reasons:	
	Lead wire is not connected to the probe and device correctly.	Ensure the probe and device are connected correctly.  To assess if the device is faulty, dampen your hand with water and a bit of salt. Firmly grip the probe, ensuring your skin covers the metal parts. Gradually increase the strength until you feel something, typically around 25.0 mA. If the LEADS alarm activates and the unit won't exceed 20.0 mA, the lead wires need replacing.
	There is a poor connection between your body and the vaginal probe.	Use a water-based lubricant such as TensCare Go Gel to improve conductivity.  Cross your legs and squeeze. This can improve the probe's connection. If this helps, the contact should improve after a few weeks of use. If not, the unit may be unsuitable for you. Consult your healthcare professional for alternative options.

No stimulation sensation	Intensity level is not high enough.	Ensure you increase the intensity sufficiently. With the probe inserted, you may need to raise it to 40.0–60.0 mA. Max power is 99 mA. Adjust the intensity until you feel it.
	Reduced sensitivity in the area being treated.	You may have reduced sensitivity due to previously damaged or desensitised pudendal nerves (this can happen in childbirth or some surgical procedures). Please consult your healthcare professional.
No sensation of one side of the probe	The probe is not positioned correctly.	It's impossible for one side of the probe to malfunction while the other operates correctly. Instead, you might experience a stronger sensation on one side. To address this, try adjusting the probe's position slightly, making sure the metal plates face outward toward the hips.

Sudden change in sensation

If you stop the stimulation and continue a few minutes later, the signal will feel quite a lot stronger.

Always return the intensity to zero after disconnecting the leads or electrode pads.

If you are unable to resolve the issue, please contact our customer service team:

Email: [customer.service@tenscare.co.uk](mailto:customer.service@tenscare.co.uk)

Phone: +44 1372 723434

**Note:** Unicare has no essential performance characteristics. Device failure will not cause unacceptable clinical risk for the user.

## 4. Device Care

### 4.1. Cleaning

It is important that the probe is cleaned before using for the first time and after each use.

1. Clean with either an alcohol-free antibacterial wipe such as TensCare wipes or by washing with warm soapy water.
2. Rinse in clean water and dry thoroughly.
3. Return the unit to the storage pouch.

**Warning:** Do not immerse the probe in a liquid.

Clean the device and lead wires at least once a week by wiping with a damp cloth and solution of mild soap and water. Wipe dry.

- Do not immerse the device in water.
- Do not use any other cleaning solution than soap and water.

## 4.2. Storage

Between uses it is important to store the electrodes in their resealable plastic bag with the bag sealed. This will help extend the life of the electrodes.

Keep the device and all accessories in the storage pouch between uses.

### Storage life

- The device has no fixed shelf life.
- Unused batteries have a nominal shelf life of 3 years but will usually last longer than this.

## 4.3. Charging Unicare

Unicare is powered by a built-in rechargeable Li-ion battery.

A charging adapter is included in the kit.

When the batteries are low and the battery icon is on the display charge your device at this stage.

 **Warning:** Only use the charging adapter supplied. Use of other chargers could be hazardous and will negate the warranty.

To charge the battery:

Connect the charger to the unit and plug it into a mains socket. A charging screen will be displayed, showing the battery percentage level. Charge the device until the battery level is 100%.

 **Note:** The device cannot be used while being charged.

## Battery life and maintenance

The battery should last at least 15 hours at 50mA, 300 $\mu$ s, 50Hz.

To fully charge the battery it will take about 2 hours.

 **Note:** The device should be charged at least once every 3 months to ensure the battery life is maintained.

 **Note:** Charging voltage: DC 5V, charging current: 90mA.

### Battery Warnings

- Remove batteries from the device if it is unlikely to be used for a long period. Some types of batteries may leak corrosive fluid.
- Do not pierce, open, disassemble, or use in a humid and/or corrosive environment.
- Do not expose to temperatures over 60°C(140F).
- Do not put, store or leave near sources of heat, in direct strong sunlight, in a high temperature location, in a pressurized container or in a microwave oven.
- Do not immerse in water or get wet.
- Do not short-circuit.
- Do not connect the device unless the battery cover is in place.
- If battery leakage occurs and comes in contact with the skin or eyes, wash thoroughly with lots of water and immediately seek medical attention.
- Keep batteries out of the reach

of children to prevent them from swallowing them by mistake. If swallowed by child, contact doctor immediately.

- Never attempt to recharge an alkaline battery. Risk of explosion.
- Do not mix old, new or different types of batteries as this may lead to battery leakage or low battery indication.
- Disposal: Always dispose of batteries and device responsibly according to local government guidelines. Do not throw batteries onto a fire. Risk of explosion.

#### 4.4. Probe and lead wire care

 **Caution:** The stainless steel in the probe's metal plates can contain some Nickel. This could cause a reaction if you have a Nickel allergy.

- Replace the probe every 6 months to maintain proper hygiene.
- Do not pull on the lead wires of the probe. This may cause damage.
- The probe is intended for single patient use only. Do not share the probe with anyone else. Improper treatment or cross-infection may occur.
- It is important the probe is cleaned after each use. Ineffective cleaning may lead to irritation or infection.
- To ensure safety and effectiveness we recommend using TensCare probes and electrodes only.

- The life of the lead wires depends greatly on use. Always handle the leads with care. We recommend replacing the lead wires regularly (about every 6 months).

#### 4.5. Electrode care

The electrodes are reusable but for single patient use. The electrodes should provide up to 20 uses. This may vary depending on skin condition, humidity and storage between uses.

To maximise the life of the electrodes, always ensure the electrodes are replaced on to their plastic liner and stored and sealed in the plastic bag. When the electrodes dry, this may negatively impact the performance of the treatment.

The adhesive in the electrode is a peelable hydrogel (water based). To restore the tackiness of the electrode, add a tiny drop of water and spread it around. Avoid adding too much, as it will make the electrode too soft. If this occurs, place the electrodes sticky side up in a refrigerator for a few hours to regain adhesiveness.

Replace the electrodes when they lose their stickiness. Poor connection may cause discomfort and skin irritation.

To ensure safety and effectiveness we recommend using TensCare electrodes only.

#### 4.6. Replacement parts

The following replacement parts are compatible with the Unicare and may be ordered from TensCare at [www.tenscare.co.uk](http://www.tenscare.co.uk) or +44(0) 1372 723434.

X-VP	Liberty vaginal probe (28mm diameter)
X-VPL	Liberty loop vaginal probe (32mm diameter)
X-VPM	Liberty plus vaginal probe (32mm diameter)
X-AP	Liberty mini vaginal probe (24mm width)
X-VPLG	Gold liberty loop vaginal probe (32mm diameter)
X-PR13	Liberty fit anal probe
K-GOI00	Go Gel water-based lubricant

E-CM5090	50x90mm electrodes (x2)
L-CPT	Lead wire
X-BX-UR	Belt clip
X-UNICH	Charging adapter
X-MULTI-UK	UK socket
X-MULTI-EU	EU socket
X-MULTI-USA	US socket

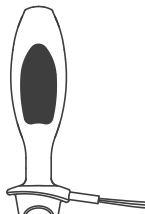
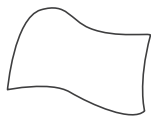
**Warning:** Do not use this device with any probe other than those recommended above. Probes with smaller surface area may cause tissue irritation.

## 5. Technical Specification

Waveform	Symmetrical rectangular
Voltage	50V zero to peak across 500Ω, Constant voltage over 470-1500 Ohm, Constant current over 160-470 Ohm
Amplitude (over 500 Ohm load)	99.0 mA zero to peak +ve in 99 steps +/- 10%
Frequency range	10-110Hz
Pulse width range	100-300μs
Output plug	Fully shielded: touch proof
Channels	Single channel
Power supply	Lithium-ion battery: 1000mAh, 3.7Wh, 3.7V
Weight	104g
Dimensions	120 x 60 x 20 mm

Safety Classification	Internal power source. Designed for continuous use. No special moisture protection.
Service life	Device: >5 years, Probe: 6 months
Applied part	Vaginal probe, skin surface electrodes
Applied part classification	Type BF
IP rating	IP22
Environmental Specifications	
Operating conditions	Temperature range: 5 to 40°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa
Storage conditions	Temperature range: -25 to +60°C Humidity: 15 to 90% RH non-condensing Atmospheric pressure: 700 hPa to 1060 hPa

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# Thank you for using Unicare...

At TensCare, we strive to provide the best possible products and services. We value your feedback and continuously work to enhance our offerings. Understanding how our products benefit you is important to us. If you have any insights or suggestions, please get in touch.

## Contact us

-  TensCare Ltd.  
9 Blenheim Road,  
Epsom, Surrey,  
KT19 9BE, U.K.
-  +44 (0) 1372 723434
-  [www.tenscare.co.uk](http://www.tenscare.co.uk)
-  [customer.service@tenscare.co.uk](mailto:customer.service@tenscare.co.uk)

## Follow us

-  TensCare Ltd
-  @tenscareltd
-  @TensCareLtd
-  TensCare Limited

## EU Declaration of Conformity

Tenscare Ltd herewith declare that the product meets the provisions of REGULATION (EU) 2017/745 which apply to it. The medical device has been assigned to class IIa according to Annex IV of the REGULATION (EU) 2017/745. The product concerned has been designed and manufactured under a quality management system according to Annex IX, of REGULATION (EU) 2017/745. This EU declaration of conformity is issued under the sole responsibility of the manufacturer. No "Common Specification" is applicable.

[www.tenscare.co.uk](http://www.tenscare.co.uk)

**TensCare Ltd.**

9 Blenheim Road, Epsom,  
Surrey, KT19 9BE, U.K.

**EU DoC**

Complies with Medical Device  
Regulation (EU) 2017/745.

**Advena Ltd.**

Tower Business Centre,  
2nd Flr, Tower Street, Swatar,  
BKR 4013, Malta

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