# Liberty

## Vaginal Probe

**Instructions For Use**

Please read carefully

**TensCare™**

## INTENDED USE

The **Liberty vaginal probe** is a medical device designed to be used in the home healthcare environment alongside Pelvic Floor Muscle Stimulators to stimulate and train the pelvic floor muscles, helping to treat symptoms of urinary incontinence. Pelvic floor stimulation also helps to relieve symptoms of chronic pelvic pain. Additionally, it may help improve sexual intimacy by toning the pelvic floor muscles.

For use only with Muscle Stimulation equipment approved to EN 60601-1, EN60601-2-10 and EMG Biofeedback equipment approved to EN60601-1 in the treatment of 'Stress', 'Urge' and 'Mixed' and/or faecal incontinence as well as pelvic pain.

**For Women:** The **Liberty** Probe is intended for the treatment of stress, urge and mixed urinary incontinence, and pelvic pain.

## CONTRAINDICATIONS

Follow the Contraindications and Warnings supplied with your stimulator or biofeedback device.

Do not use with electrical stimulation in patients with a history of carcinoma at the site of stimulation.

## PROBE CAUTIONS

**Caution:** This probe is intended for single patient use only. Do not share your probe with anyone else. *Improper treatment or cross-infection may occur.*

**Caution:** It is important that the probe is cleaned before and after each use. *Ineffective cleaning may lead to irritation or infection.*

**Caution:** Never insert or remove the probe unless the control unit is powered OFF as insertion or removal when stimulation is active may cause discomfort or tissue irritation.

**Caution:** If tissue irritation occurs, discontinue treatment immediately. *Ask your healthcare professional for advice before continuing further treatment to prevent injury.*

**Caution:** Do not use a silicone-based lubricant on the metal plates of the probe as it may decrease the effectiveness of the electrical stimulation.

**Caution:** Electrode surface area is 8.58cm<sup>2</sup>. At high output settings on some devices, this could result in a current density greater than 2mA RMS/cm<sup>2</sup>. High current density may be associated with localised tissue damage or minor non-thermal burns. Increase intensity carefully and reduce if you experience sharp pain during use or discharge or bleeding after use.

**Caution:** The stainless steel in the probe's metal plates contain some Nickel. This could cause a reaction if you have a Nickel allergy. *Alternative gold probe specially made without Nickel are available.*

**Warning:** Do not use if the pack is opened or damaged.

## INSTRUCTIONS FOR USE

1. Clean the probe with an alcohol-free anti-bacterial wipe or with warm soapy water.
2. Connect the lead from the base of the unit to the lead in the probe. Push the pin ends firmly into the pigtail ends of the probe lead.
3. Before using the probe, you will need to visit the toilet.
4. Lubricate the metal electrode surfaces and probe tip with a water-based lubricant such as TensCare Go Gel.

 **Caution:** Do not use a silicone based lubricant on the stimulation contacts as it may decrease the effectiveness of the muscle stimulation.

5. Choose a comfortable position, such as lying down on your bed on your side with your knees raised.

 **Warning:** Ensure the unit is switched OFF before inserting the probe.

6. **Vaginal insertion:** After the wire is securely connected, insert the probe into your vagina, in the same way as you would a tampon, until only the plastic ring at the base of the probe is visible. The metal parts conduct the electrical pulse and should be in contact with the main part of the muscle at all times. The tissues close to the entrance are more sensitive, so you should avoid stimulating them. Probes with long electrodes (the metal part) that run up and down the length of

the attachment should always be inserted with the metal parts facing hip-to-hip.

7. After your training session, switch the control unit off, then remove the probe by holding the positioning end rim and gently pulling outwards.

## **CLEANING**

It is important that the probe is cleaned before and after each use. Clean with either an alcohol-free antibacterial wipe such as TensCare Wipes (see X-WIPES) or by wiping with warm soapy water. Rinse under running water and dry thoroughly and return the unit to the storage pouch.

### **Warnings:**

Do not immerse the probe in a liquid.

Keep away from children under 5 years of age. Long cable - risk of strangulation in infants.

Keep the device away from lint and dust, as long-term exposure to lint or dust may affect the socket.

Keep the device away from sunlight, as long-term exposure to sunlight may affect the rubber causing it to become less elastic and crack.

## **DISPOSAL**

Clean thoroughly and dispose of according to local regulations.

## **TECHNICAL SPECIFICATIONS**

Material: Electrode body: Plastic ABS PA-757

Electrodes: Stainless steel #304

Cables: PVC over copper tinsel conductor

“In Use” product lifetime: 6 months

## ENVIRONMENTAL CONDITIONS:

Storage: -25-+60°C, 15-90% RH, 700-1060 hPa

Operation: 5-40°C, 15-90% RH, 700-1060 hPa


Water ingress protection: IP22

The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm  $\varnothing$ , 80 mm length

The second number 2: Protected against vertically falling water drops when enclosure is tilted up to 15°.

This symbol on the device means “Refer to instructions for use”

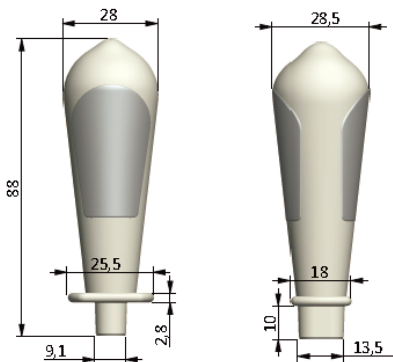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**TensCare Ltd.**
**Advena Ltd.**