# Gold Liberty Loop

## Vaginal Probe

### Instructions For Use

Please read carefully

**TensCare™**

## INTENDED USE

The **Gold Liberty Loop** vaginal probe is a medical device designed to be used in the home healthcare environment alongside Pelvic Floor Muscle Stimulators to stimulate and train the pelvic floor muscles, helping to treat symptoms of urinary incontinence. Pelvic floor stimulation also helps to relieve symptoms of chronic pelvic pain. Additionally, it may help improve sexual intimacy by toning the pelvic floor muscles.

For use only with Muscle Stimulation equipment approved to EN 60601-1, EN60601-2-10 and EMG Biofeedback equipment approved to EN60601-1 in the treatment of 'Stress', 'Urge' and 'Mixed' urinary incontinence and/or pelvic pain.

**Note:** This probe is gold-plated, making it suitable for individuals with nickel sensitivity or allergy.

## CONTRAINDICATIONS

Follow the Contraindications and Warnings supplied with your stimulator or biofeedback device.

Do not use with electrical stimulation in patients with a history of carcinoma at the site of stimulation.

## PROBE CAUTIONS

**Caution:** This probe is intended for single patient use only. Do not share your probe with anyone else. *Improper treatment or cross-infection may occur.*

**Caution:** It is important that the probe is cleaned before and after each use. *Ineffective cleaning may lead to irritation or infection.*

**Caution:** Never insert or remove the probe unless the control unit is powered OFF as insertion or removal when stimulation is active may cause discomfort or tissue irritation.

**Caution:** If tissue irritation occurs, discontinue treatment immediately. *Ask your healthcare professional for advice before continuing further treatment to prevent injury.*

**Caution:** Do not use a silicone based lubricant on the metal plates of the probe as it may decrease the effectiveness of the electrical stimulation.

**Caution:** Electrode surface area is 4.24cm<sup>2</sup>. At high output settings on some devices, this could result in a current density greater than 2mA RMS/cm<sup>2</sup>. High current density may be associated with localised tissue damage or minor non-thermal burns. Increase intensity carefully and reduce if you experience sharp pain during use or discharge or bleeding after use.

**Warning:** Do not use if the pack is opened or damaged.

## INSTRUCTIONS FOR USE

1. Connect the lead from the base of the unit to the lead in the probe. Push the pin ends firmly into the pigtail ends of the probe lead.
2. Before using the probe, you will need to visit the toilet.
3. Lubricate the metal electrode surfaces and probe tip with a water-based lubricant, such as TensCare Go Gel.

**Caution:** Do not use a silicone-based lubricant on the stimulation contacts as it may decrease the effectiveness of the muscle stimulation.

4. Choose a comfortable position, such as lying down on your bed on your side with your knees raised.


**Warning:** Ensure the unit is switched OFF before inserting the probe.

5. **Vaginal insertion:** After the wire is securely connected, insert the probe into your vagina, in the same way as you would a tampon, until only the plastic ring at the base of the probe is visible. The metal parts conduct the electrical pulse and should be in contact with the main part of the muscle at all times. The tissues close to the entrance are more sensitive, so you should avoid stimulating them. This probe should always be inserted with the metal parts facing hip-to-hip.
6. The **Gold Liberty Loop** probe has a pointer attachment which enables you to visually monitor the successfulness of your pelvic floor contractions (the probe can be used without the pointer). The pointer

demonstrates the strength and duration of each contraction, whether caused by stimulation or performed voluntarily. For example, weak contractions are defined by little or no movement in the pointer. Assemble the pointer and attach to the probe as follows: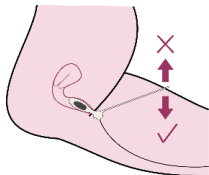

When performed correctly, the pointer will move downwards.

- When performed incorrectly, the pointer will move upwards.

7. After your training session, switch the control unit off, then remove the probe by holding the positioning end rim and gently pulling outwards.

## CLEANING

It is important that the probe is cleaned before and after each use. Clean with either an alcohol-free antibacterial wipe such as TensCare Wipes (see X-WIPES) or by wiping with warm soapy water. Rinse under running water and dry thoroughly and return the unit to the storage pouch.

### Warnings:

Do not immerse the probe in a liquid.

Keep the device away from lint and dust, as long-term exposure to lint or dust may affect the socket.

Keep the device away from sunlight, as long-term exposure to sunlight may affect the rubber causing it to become less elastic and crack.

Keep away from children under 5 years of age. Long cable - risk of strangulation in infants.


## TECHNICAL SPECIFICATIONS

Material: Electrode body: Plastic ABS PA-757

Electrodes: Gold plated (15K) Stainless steel #304

Cables: PVC over copper tinsel conductor

“In Use” product lifetime: 6 months

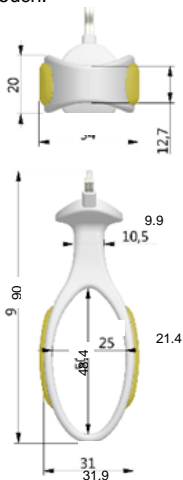## **ENVIRONMENTAL CONDITIONS:**

Storage: -25+60°C, 15-90% RH, 700-1060 hPa

Operation: 5-40°C, 15-90% RH, 700-1060 hPa

Water ingress protection: IP22

The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm  $\varnothing$ , 80 mm length

The second number 2: Protected against vertically falling water drops when enclosure is tilted up to 15°.

This symbol on the device means “Follow operating instructions”

To report unexpected operation or events contact Tenscare Ltd.  
Please find our details on the final page of this insert.

## **DISPOSAL**

Clean thoroughly and dispose of according to local regulations.

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**TensCare Ltd.**
**Advena Ltd.**